

User's Manual



Win-Track


medicapteurs
PODLOGIE, POSTUROLOGIE ET RÉÉDUCATION
« Les Espaces de Balma »
18 avenue Charles de Gaulle - Bât. 34
31130 BALMA
Tel : 33 (0) 562 571 571
www.medicapteurs.fr

Foreword

We appreciate your trust in the MEDICAPTEURS Company and we would like to thank you for choosing our product.

The "WIN-TRACK Software" has been designed to be user friendly and only requires basic computer skills. The friendliness of the software however, does not replace **the necessity of reading this manual** to guide you step by step through the procedures to properly use the "WIN-TRACK Software", and it will also help you save valuable time.

In addition to this manual, you will find the **Installation Manual** for the platform and the software, on your Medicapteurs CD or USB key.

Enjoy your reading and welcome to the privileged "WIN-TRACK" users' Club.

Contents

CONTENTS.....	5
WARRANTY.....	7
HISTORICAL BACKGROUND.....	9
IMPORTANT NOTICE	10
WIN-TRACK SOFTWARE START UP	11
EXIT WIN-TRACK SOFTWARE.....	11
WIN-TRACK SOFTWARE NAVIGATION	12
MENU BAR.....	13
MENU "FILES"	13
MENU "UTILITIES"	14
MENU "STATISTICS"	15
MENU "CONFIGURATION"	16
WELCOME PAGE.....	19
SCHEDULE.....	20
SCHEDULE PARAMETERS	20
SCHEDULE BROWSING	21
MAKING AN APPOINTMENT	22
VISIT FLOW CHART	23
"PATIENT" TAB	24
TOOLS BAR	24
CREATING A NEW PATIENT FILE.....	25
SEARCHING FOR AN EXISTING PATIENT FILE	25
MODIFYING AN EXISTING PATIENT FILE.....	25
"EXAM" TAB.....	26
TOOL BAR	26
PREVIOUS EXAMS	27
ATTACHING DOCUMENTS TO AN EXAM.....	27
CREATING A NEW EXAM	28
SEARCHING FOR AN EXISTING EXAM	28
MODIFYING AN EXAM FILE	28
"ACQUISITION" TAB.....	29
TOOL BAR	30
CREATING A MEASUREMENT (ACQUISITION).....	31
STATIC ACQUISITION	32
BASIC DYNAMIC ACQUISITION (ONE GAIT CYCLE)	33
MULTIPLE DYNAMIC ACQUISITIONS (MULTIPLE STEPS)	35
POSTURAL ACQUISITION	37
DETAILED STATIC ANALYSIS.....	39
TOOL BAR	39
FOOT PRINT ANALYSIS.....	40
"STANDARD" VIEW	41
"ISOPRESSURE" VIEW	42
"3D" VIEW	43
"MEASUREMENTS" OPTION.....	44
"LINE OF BESSOU"	46

DETAILED DYNAMIC ANALYSIS	48
TOOL BAR	49
FOOT PRINT ANALYSIS	50
MULTIPLE VIEWS	53
GRAPHS – DYNAMIC	54
TOOL BAR	55
SELECTING A GRAPH	56
GRAPHIC OPTIONS	56
FEET SYNTHESIS	57
COMPARING 2 MEASUREMENTS	58
MEDIO-LATERAL ANALYSIS	59
DETAILED POSTURAL ANALYSIS	60
TOOL BAR	61
FOOT PRINT ANALYSIS	62
THE DIFFERENT VIEWS	65
"MOZAIC" VIEW	66
GRAPHS - POSTURAL ANALYSIS	67
GRAPH SELECTION	67
GRAPHS DISPLAY OPTIONS	68
PARAMETERS	68
MEASUREMENTS COMPARISON AND ROMBERG QUOTIENT	69
ACQUISITION	69
ANALYSIS	69
"COMPARISON" TAB	70
GENERAL INFORMATION	70
UP TO 4 MEASUREMENTS COMPARISON	71
"REPORT" TAB	72
TOOL BAR	72
CREATING A PERSONAL NOTE	73
INSERTING AN IMAGE	73
"VIDEO" TAB	74
TOOL BAR	74
SIMULTANEOUS RECORDING VIDEO / PODOMETRY	75
POST-SYNCHRONIZATION OF THE VIDEO WITH THE DYNAMIC FOOT PRINT	75
"MAILING" TAB	76
PRINTING	78
SCALE 1 PRINTING	79
EXPORTING DATA	80
PATIENT DATABASE BACK UP	81
PATIENT DATABASE BACK UP	81
RELOADING A BACKED UP PATIENT DATABASE	81

Warranty

For a period ending on the date of the twelfth month anniversary of the date of delivery (such period being the "Warranty Period"), the equipment, used under normal operating conditions and not subject to misuse or abuse, shall be free from defects in material, workmanship and design and shall operate and conform to the performance capabilities, specifications, functions and other descriptions and standards as described in the specifications.

During the Warranty Period and subject to the exclusions below, Manufacturer at its option and expense, shall promptly repair or replace equipment that does not conform to the warranty. The procedure for effecting warranty replacement or repair will be as specified by Manufacturer. Purchaser shall ship Equipment for warranty replacement or repair to Manufacturer at Purchaser's expense, and Manufacturer shall ship repaired or replaced equipment to Purchaser at Manufacturer's expense.

For proper shipping of the equipment to MEDICAPTEURS for repair or maintenance, Purchaser shall include complete sender's information, the equipment serial number and a brief description of the problem.

Purchaser shall use preferably the original shipping package. If it is not available, Purchaser shall contact a professional shipping service company which will assist Purchaser with proper packaging instructions and materiel.

Limitations of warranty and disclaimer

The Software:

The Software is and shall remain the exclusive property of Manufacturer. This license confers no title or ownership in the Software and is not a sale of any rights in the Software. No license other than that specifically stated herein is granted to Purchaser, and Purchaser shall have neither right to sublicense the Software nor any right under any patent, trademark, copyright, trade secret or other intellectual property of Manufacturer other than that granted by this Agreement.

Manufacturer grants Purchaser, a personal, non-exclusive, non-transferable, perpetual license to use the Software in the product under the terms and conditions stated herein. If the license is transferred by the Purchaser to a third party without the company's written consent and approval, the Purchaser and the Third Party shall be held jointly responsible for payment to the Manufacturer of a minimal compensation of 20 times the price of the software, without prejudice as to any other damage and interest. The software travels at the Purchaser's risk and expenses.

Purchaser shall not make any modifications to the Software or remove any proprietary, regulatory or safety notices of Manufacturer in or on the Software. Purchaser further agrees not to reverse engineer, decompile or otherwise reverse assemble any portion of the software or create derivative works based upon the Software.

This Software Agreement does not grant Purchaser any rights in connection with any trademarks or service marks of Product.

Manufacturer warrants to purchaser that, in case of any software it has identified at its own property, the software will substantially conform to the applicable Manufacturer's specifications in the date of installation and for a period that is consistent with the warranty offered herein. Manufacturer's sole obligation under this agreement is limited to responding to purchaser's contact and to using commercially reasonable efforts to correct reported problems by supplying purchaser with a corrected version of the Software.

Warranty

Copyright:

All title and intellectual property rights in and to the content that may be accessed through use of the Software is the property of the respective content owner and may be protected by applicable copyright or other intellectual property laws and treaties. This agreement grants Purchaser no rights to use such content except as accessed through the Software. If the Software contains documentation that is provided only in electronic form, Purchaser may print copy of such electronic documentation.

Disclaimer:

Manufacturer does not warrant that operation of its Software will be uninterrupted or error free, or that any defects in the Software are correctable or will be corrected; or that functions contained in the Software will operate in the combinations which may be selected for use by purchaser to meet Purchaser's requirements. Manufacturer's warranty and seller's obligations will be void if the Software is modified without the prior written consent of Manufacturer.

Except for the express limited Software warranty for the Software, Manufacturer and seller expressly disclaim and exclude all other warranties, whether express, implied in law or otherwise to the fullest extent permitted by law, and the above limited Software warranty is in lieu of all obligations and liabilities on the part of the Manufacturer for damages arising out of or in connection with the Software, including without limitation any warranty of merchantability, satisfactory quality or fitness for a particular purpose.

Exclusions: Seller and Manufacturer shall have no obligations with respect to any defects, damage, or non-conformances of Equipment that result from: improper installation, operation or maintenance; unauthorized repair; tampering; any use of the Equipment not in conformance with the relevant Equipment specifications or from accidents, or "acts of God"; or other such occurrences after delivery. If any Equipment returned by Purchaser for repair or replacement is not defective or nonconforming or contains a defect or nonconformance caused by a factor outside the scope of Manufacturer's warranty, Manufacturer shall bill Purchaser at Manufacturer's normal service rate for services performed with respect to such defective returned Equipment or for "no fault found" with respect to returned Equipment which is not defective.

LIMITATION ON WARRANTY:

MANUFACTURER AND SELLER MAKE NO WARRANTIES REGARDING THE EQUIPMENT TO BE PROVIDED HEREUNDER OTHER THAN THE EXPRESS WARRANTIES IN THIS SECTION AND THERE SHALL BE NO IMPLIED WARRANTIES, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR NONINFRINGEMENT. THE SOLE REMEDIES OF PURCHASER FOR ANY BREACH OF WARRANTY ARE THE REMEDIES SET FORTH IN THIS SECTION. MANUFACTURER AND SELLER SPECIFICALLY DISCLAIMS ANY AND ALL WARRANTIES TO CUSTOMERS OF PURCHASER.

Limitation of Liability: In no event shall Manufacturer or its Affiliates, Seller or suppliers be liable for indirect, special, incidental, consequential, punitive or other damages (including lost revenue, lost profit, lost data, downtime costs,), arising out of the use, inability to use, or the result of the use of the Equipment whether based on warranty, breach of warranty, contract, tort, or other legal theory, and whether or not advised of the possibility of such damages. In no event shall Manufacturer or its Affiliates, Seller or suppliers be liable to Purchaser for any damages, however based, in excess of the amount paid by the Purchaser to Seller for the Equipment.

Historical background

The first recording of plantar support was performed in 1880 through the work of MAREY and DEMENY.

As early as 1947, SCHWARTZ used piezo-electric sensors positioned under the plantar surface of the feet to perform a dynamic gait analysis.

In 1962, CARLSON and TAILLARD further the studies on dynamic analysis.

In 1966, significant studies were conducted using the electro podography method established by INSERM research institute in Montpellier, France. It studied the static and dynamic behavior of plantar support with sensors/insoles and a dynamometric mat.

In 1983, the studies performed by Professor RABISCHONG led to the development of sensitive insoles using « artificial skin » technology (Registered by ANVAR/CNRS).

Simultaneously, Mr. CLOT's and Mr. ESTEVE's studies from the research labs of the LAAS/CNRS in Toulouse, France resulted in the design of an **Electronic Podometer**: a Platform for the Analysis of Static and Dynamic Plantar Supports.

In **1985**, the company MIDI-CAPTEURS manufactured and commercialized the **Electronic Podometer PEL 38**.

PEL 38 became the reference not only for the quality level of the analysis results but also for the design of the software and positioned MIDI-CAPTEURS as the European leader in the field of Electronic Podometers.

The **WIN-POD** system was born in 2003, using the latest calibrated sensors' technology for continuously improving performance.

In **2004**, MEDICAPTEURS launched the **FUSYO platform**, and became the only manufacturer of a system integrating an electronic podometer into a posturology platform built according to APE specifications.

With the introduction of **WIN-TRACK** in 2009, MEDICAPTEURS strengthened its leadership position in the design and manufacturing of podiatry, posturology, and biomechanical analysis platforms, serving more than 5000 customers and numerous renowned distributors, resulting in a constant progression in the global market.

IMPORTANT NOTICE


Your "WIN-TRACK Software" integrates a patient database (**patient files**) which contains private health information related to the patients and their specific measurements.

In order to avoid losing the studies performed with your software, it is **necessary to perform a BACK UP to save the patient database on a regular basis**; in fact any technical problem on your computer may affect and destroy such valuable data.

We highly recommend a minimum of one Back Up on a weekly basis on an external drive, CD RW, etc. (see page 81).

WIN-TRACK Software Start Up




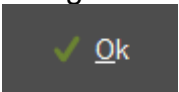
- ✓ Launch the "WIN-TRACK Software", by double clicking on the icon 
- ✓ Within a few seconds, a window will open asking for the user's password :
Note: this security option is now mandatory for software utilizing confidential patient information.

The default user in the software is: "ADMIN" which does not require a password. To create your own user's name and password please refer to page 18.


- ✓ Click  to start the program.

Exit WIN-TRACK Software

With the « Exit » Key:

- ✓ Click on the  located in the upper right corner of the screen.
- ✓ Confirm your action by clicking on .

With the "File Menu"

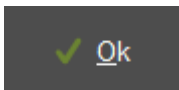- ✓ Click on "File".
- ✓ Select "Quit".
- ✓ Confirm by clicking on .


WIN-TRACK Software Navigation

Title Bar




It includes the name of the software, the patient's last name and first name, the exam date and time of the recording.

Menu Bar

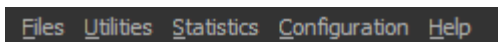

It contains the menu: **Files – Utilities - Statistics - Configuration - Help.**

Selection Bar

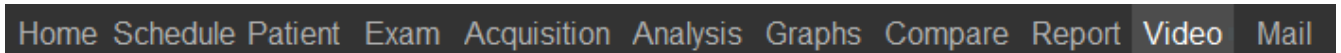

It allows access to all the phases of a podiatry assessment.
The typical process is done by clicking on the tabs from the left to the right.

Tool Bar



It includes the icons for quick access to the software functions.

Several tool bars are available according to the displayed screens.

Menu Bar


Menu "Files"

This menu can be used for creating or changing pre-established lists which can be opened when completing the patient file and the exam file.


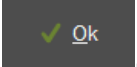
Available Lists

- Correspondents
- Clinical exam
- Complaints
- Professions


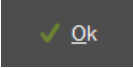
Adding Data To The List

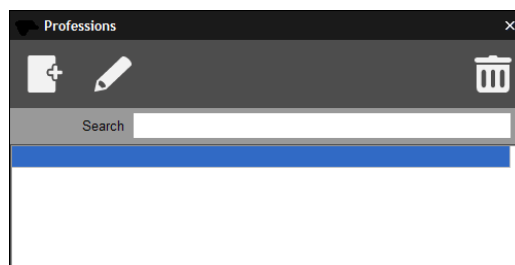

For example: For the above mentioned lists, perform the following operations:

- ✓ Click on .
- ✓ Insert the text (Start with an upper case!)
- ✓ Click on .

Deleting Data From The List

- ✓ Click on the item to erase
- ✓ Select .
- ✓ Confirm by clicking .

Special case: clinical exam

For the parameters of the clinical exam, you can modify each list as written above but also the name of the parameter. So you can entirely personalize the clinical exam.


Menu Bar

Menu "Utilities"

Exporting and importing exams


Exams can be imported or exported between "WIN-TRACK Software" users.
Click on the option you selected and follow instructions.

Mass Mailing

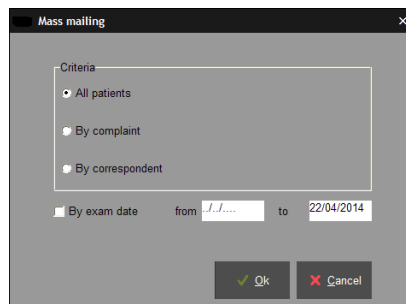


The « Mass Mailing » function allows database selection according to specific categories:

- By exam date
 - By complaint
 - By correspondent
- ✓ Choose your type of selection by clicking on exam date or a time range of exam dates.

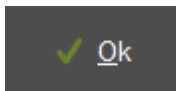

- ✓ Click and the following window will open :

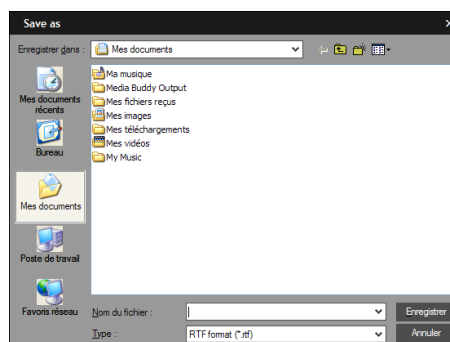

- ✓ Save your data as a text document :


To open the selected listing in a table format :

- ✓ Open a table (Excel for example).
- ✓ Load the table in a text format ".txt".
The lines and columns will be created automatically in your mail message.

Menu Bar

Menu "Statistics"

This menu allows the user to graph or list data by pathologies, practitioners, users, foot size, professions, and by gender.

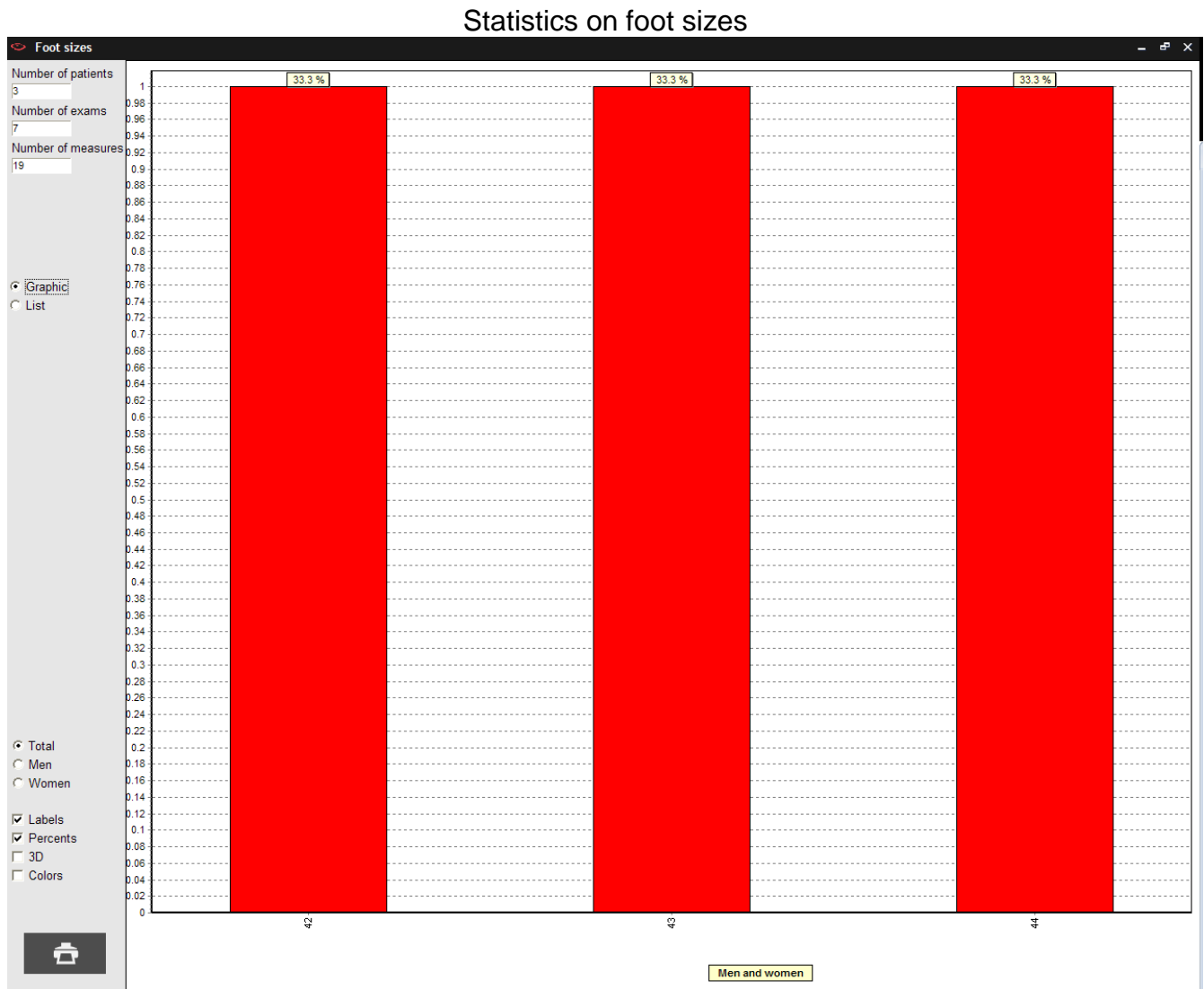


Menu Bar

Menu "Configuration"

Parameters


The Parameters window includes several tabs

- The "**General**" tab allows to modify the following parameters :

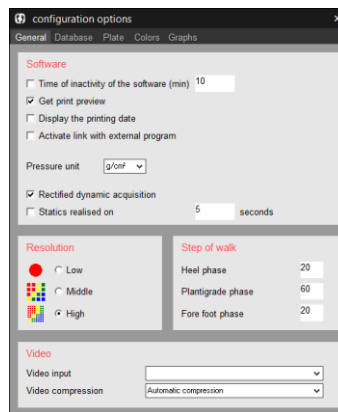

- **Software Standby Mode** : Allows you to set a timer for the software to go on standby mode. The password is required to access the software when leaving the standby mode.
- **Print preview** : Activated or not.
- ✓ **Activation of Links with External Software** : Allows you to share patient data and recordings between "WIN-POD Software" and other software (example : RM informatics).
- ✓ **Rectified dynamic acquisition** : Allows you to automatically rectify a dynamic mesure at the end of an acquisition.
- ✓ **Statics average** : Allows you to create a single static image using a static acquisition's measures, during a defined length of time (default : 5 secs).
- ✓ **Resolution** : Allows you to modify the cartography's type of visualization.
- **Phases of Gait** : Personalized norms of the three phases of the gait cycle (see page 55): you need to enter the relative timing of each phase of the cycle (in %) Heel Strike, Plantar, and Toe Off (by default: 20, 60 and 20%).
- **Measuring System** : Allows you to choose between Kpa and g/cm².

After completing your selection, the following window opens :

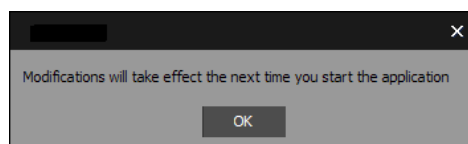


Exit "WIN-TRACK Software" and reopen it for your selections to be activated.

Menu Bar


Menu "Configuration"

Parameters


- "Data Base" tab

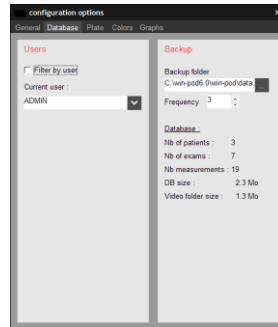

- **Users** : When more than one person uses the software, the database can be filtered according to the end user (practitioner, clinic, assistant, etc.) Select the "Filter by user" icon and then select the particular user, the software will only access the selected user database.
- **Saving** : Allows to establish database back up folder and saving frequency (based on the number of times the user's has used the software)
- "Plate" tab is reserved for hardware maintenance.
- "Colors" tab is used to redefine the screen and printing colors used in the software.

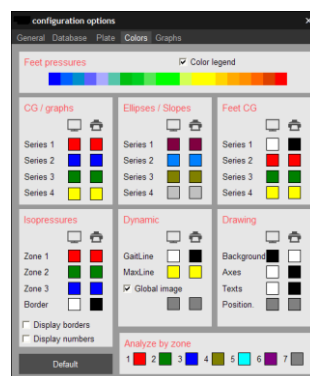

To change one color, click on color to be changed. A color palette appears on screen. Click on the desired color then validate.

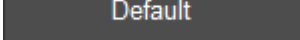Note : Selecting  will reset all colors to the software original colors.

- "Graphs" tab allows establishing preferences and parameters for visualizing graphs.


Menu Bar

Menu "Configuration"

Medical Office

Enter your office contact information and logo to be displayed at the top of printed documents.

Please perform this procedure when using the software for the first time.

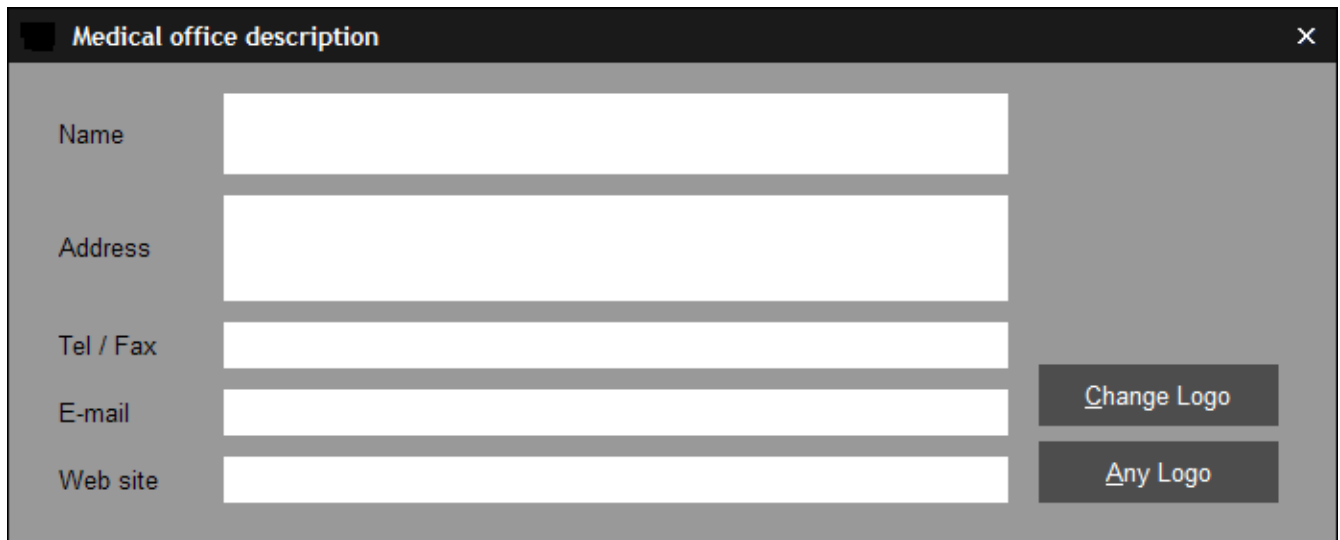

The screenshot shows a window titled "Medical office description" with a close button (X) in the top right corner. The window contains five input fields for text entry, each with a label to its left: "Name", "Address", "Tel / Fax", "E-mail", and "Web site". To the right of these fields are two buttons: "Change Logo" and "Any Logo".

Right of Access

The Software Administrator is the only person allowed to access this function. This function allows the administrator to set up other practitioners/users and their respective passwords.


More than one user can work on the same system (clinic setting). This function needs to be performed when using the software for the first time or when changes in users are necessary.

Language

Select the language you want and the software will translate automatically.

Welcome Page

Upon launching the software, the welcome page opens up so you can quickly access the main software functions. The daily appointment list will also be found.

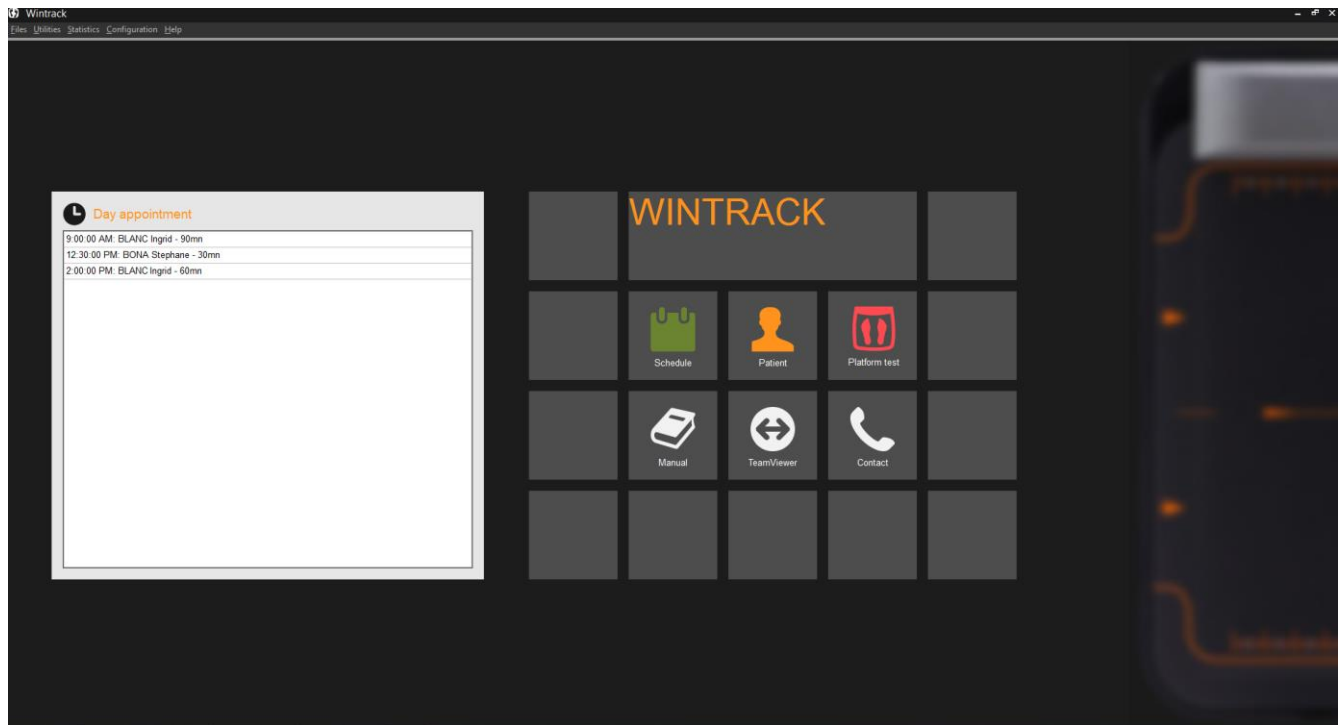

You have direct access to :



Schedule.



Patient file.



Live plate testing.



User manual



Secure software, upon selection, allows for remove access of your computer by our maintenance team.




MEDICAPTEURS' contacts list.



If W-inshoe is installed on the computer, this button appears and allows you to start it from your software.

Schedule

The scheduling screen can be selected on the welcome page. The tabs allow to manage the appointments schedule.

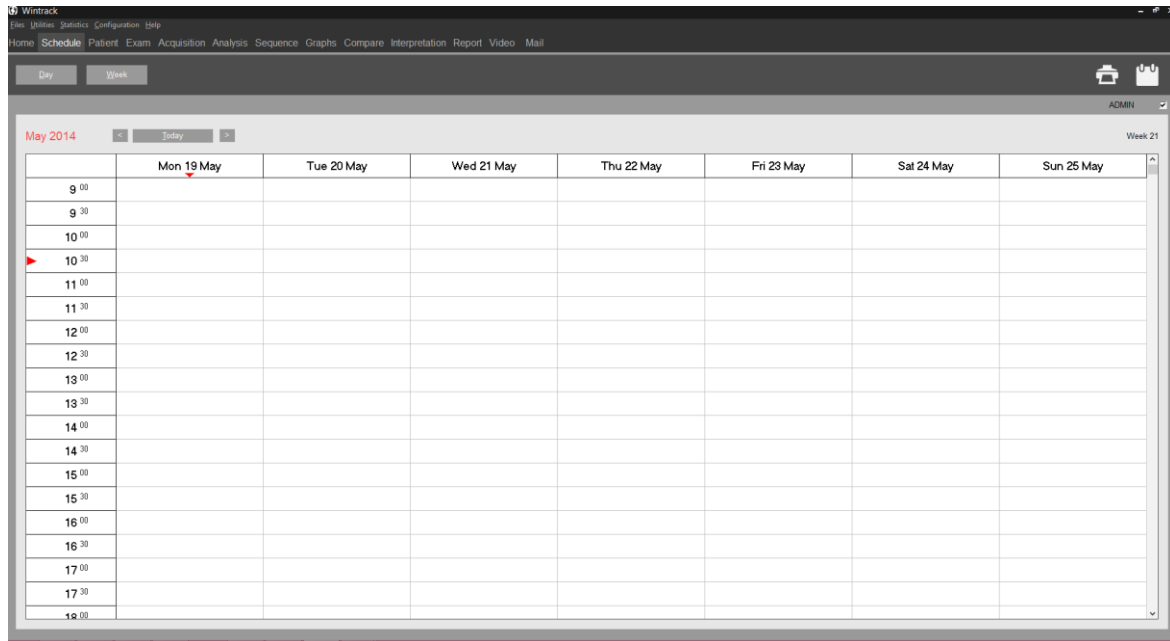


Schedule parameters

Configuration -> Schedule allows setting the scheduling parameters.

- The tab « Time slots » allows defining work days and daily schedule time slots.


By clicking on **Personalized time slots**, you can define the time slots for each day of the week.

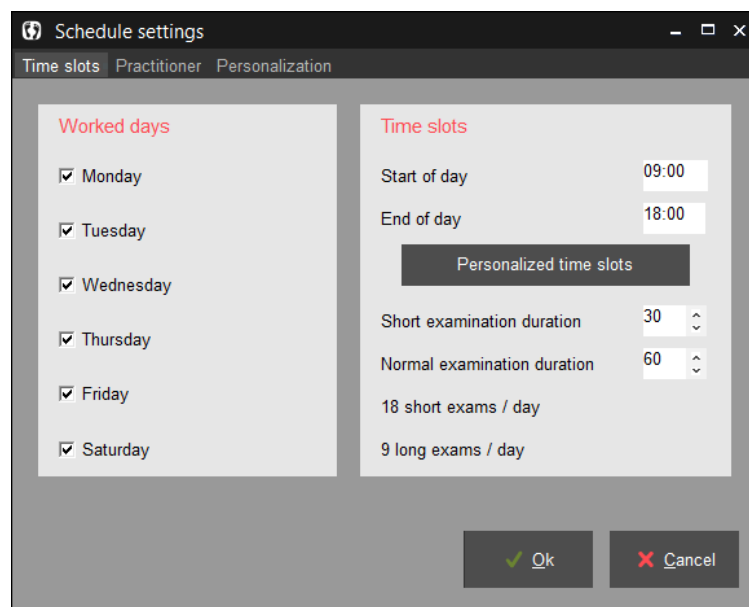




Schedule


Schedule parameters

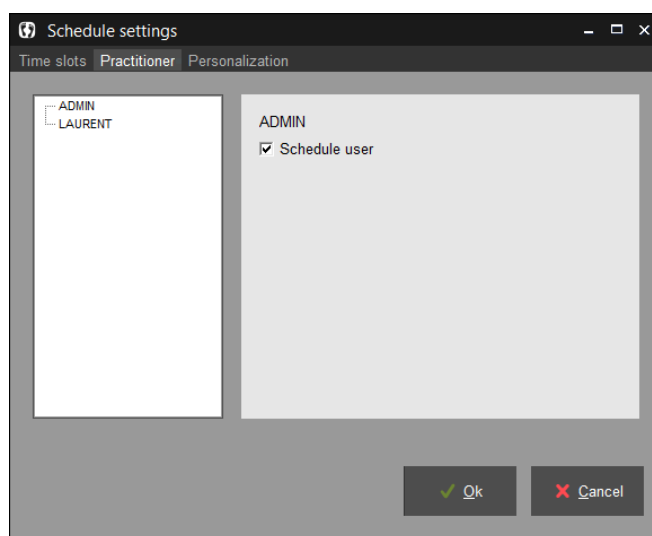- The tab « Practitioner » allows to manage the schedule for multiple software users (maximum 4 users).
It is mandatory to select the “Schedule user” icon for the user to appear on the schedule.




Schedule browsing

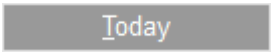You can display the schedule per day, week, or month. When the display mode is selected you can browse on the calendar by using the  and  and return to the current day by clicking on .

The icon  allows to display a specific day selected in the calendar.

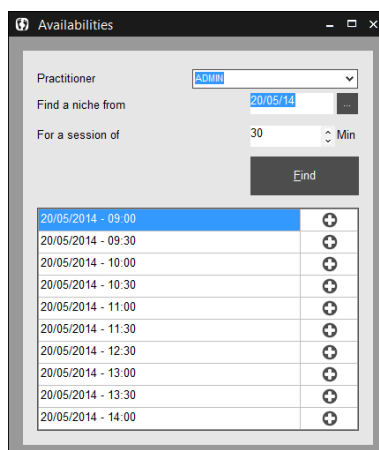


And by clicking on , you can find available times starting at a specific date.




Schedule

Making an appointment

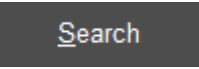After selecting the day, double click on the time slot you want. The following window opens :

- ✓ The patient selected is the currently active patient in the software by default. You can select another patient by clicking on  or by creating a new one by clicking on . If you create a new patient, you must also enter mandatory information necessary for the patient visit (refer to page 25, « Creating a new patient file »).
- ✓ Complete the required fields.
- ✓ Click on  to validate the appointment.
- ✓ The color for the appointment on the schedule varies according to the type of procedure : Clinical exam in green, Patient care in blue, Providing orthotics in yellow, Check up in gray, and Personal in red.

Visit Flow Chart

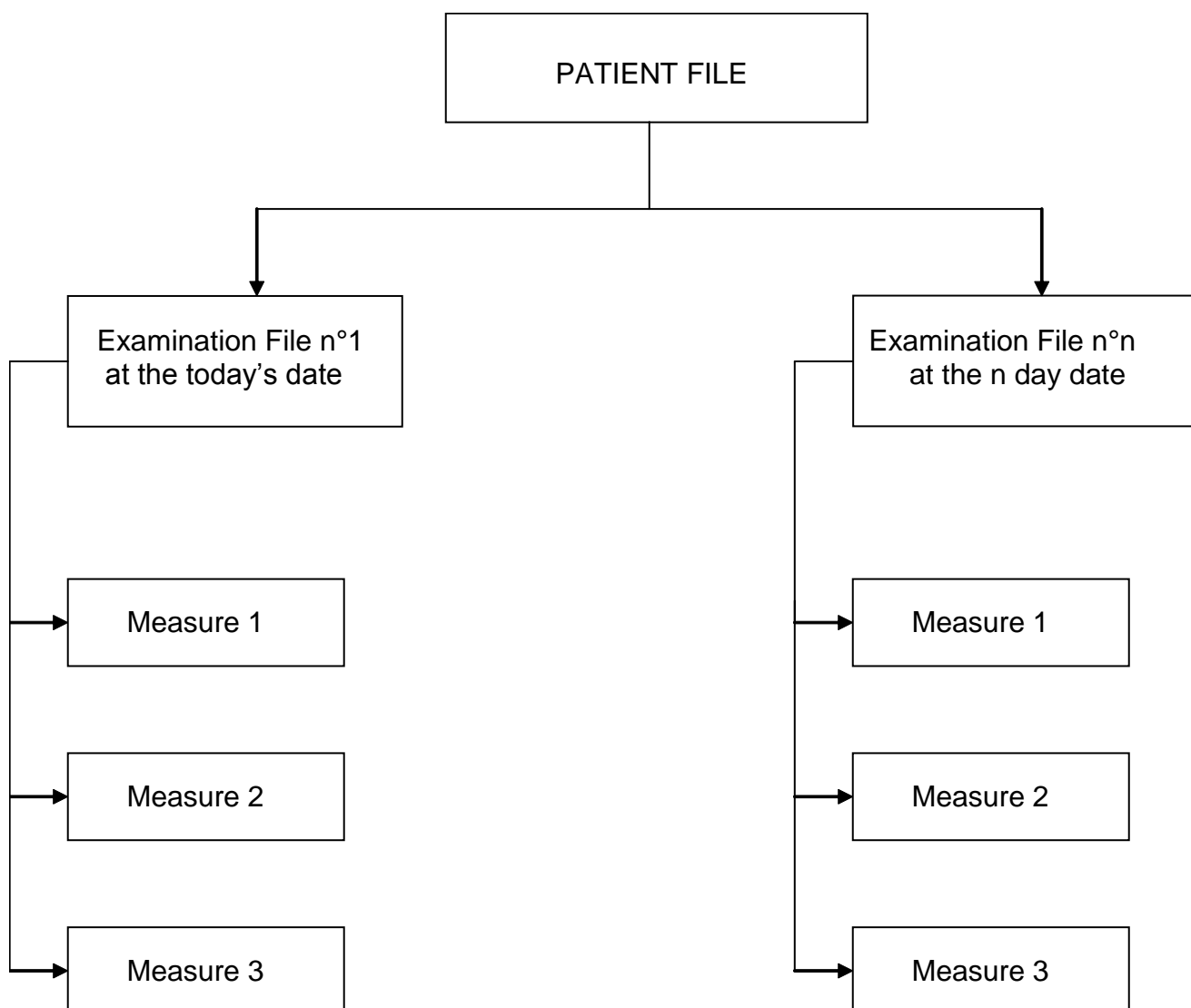

An assessment includes the following phases:

- 1) All the data from the **Patient File** including the patient's personal information and measurements.
- 2) All information from the **Exam File** related to the type of exam and patient's pathology.
- 3) Selecting the type of **Measurements** to be performed: static, dynamic or postural measurements. More than one measure can be done through one exam.
- 4) Recording the selected measurements on the hard disc through **Acquisition**.
- 5) **Analysis** of the measurements.

"Patient" Tab

The screenshot shows the Wintrack - TEST PODO application window. The menu bar includes Files, Utilities, Statistics, Configuration, and Help. The main menu has tabs for Home, Schedule, Patient (selected), Exam, Acquisition, Analysis, Compare, Report, Video, and Mail. Below the menu is a toolbar with icons for adding a patient, searching, editing, validating, canceling, printing a patient file, printing the entire list, and deleting. The Patient tab is active, displaying a form for patient data. The form is divided into two main sections: Patient and Contact. The Patient section includes fields for Created on (09/04/2010), By (ADMIN), Name (TEST), First name (PODO), Birth date (03/02/1960, 54 years), Birth place, Sex (M), Profession, Foot size (44), Height (180 cm), and Weight (92 Kg). The Contact section includes fields for Address, City code (31500), City (Toulouse), Phone (Landline, Mobile, Office), and E-mail. There is also a Note section with a text area.

Patient Folder 6

Created on 09/04/2010 By ADMIN

Name TEST

First name PODO

Birth date 03/02/1960 54 years Birth place

Sex M Profession

Foot size 44, Height 180 cm Weight 92 Kg

Contact

Address

City code 31500 City Toulouse

Phone

Landline

Mobile

Office

E-mail

Note

Tools Bar



New : Creating a new patient file.



Search : Search the database for an existing patient file.



Modify : Modify an existing patient file.



Validate.



Cancel.



Print a patient file.





Print the entire patients listing.




Permanently delete a patient file and all measurements.


"Patient" Tab

Creating a new patient file

- ✓ Select .
- ✓ Enter patient data.
Mandatory information fields will appear in blue: Name, First Name, Birth Date, Foot Size, Weight and Height.
- ✓ Use pre-defined list (see « Files » menu) for the fields where a down arrow key is present ▼ "Gender" and "Profession".
- ✓ Once the patient file is completed, click on .

Searching for an existing patient file


- ✓ Click on  to access the patient list :

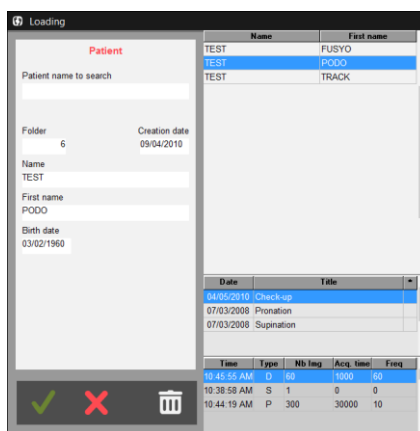



Name	First name
TEST	FUSYO
TEST	PODO
TEST	TRACK

Date	Title
04/05/2010	Check-up
07/03/2008	Pronation
07/03/2008	Supination

Time	Type	Nb. leg	Acq. time	Freq.
10:45:55 AM	D	66	1600	60
10:38:58 AM	S	1	0	0
10:44:19 AM	P	300	30000	10

- ✓ Type the first letters of the name in the field «Patient name to search ».
- ✓ Click on  or double-click directly on the patient's name.

Modifying an existing patient file

- ✓ Click on .
- ✓ Modify the concerned fields.
- ✓ Click on  to validate changes.

"Exam" Tab

An exam file consists of all the measurements performed (acquisitions) for a given patient, **on a given date**.

A new exam file must be created at each patient visit.

An exam may contain an extensive number of measurements (acquisitions) from the same exam type or different types (static, dynamic, or postural measurements).

Tool Bar



New : Creation of a new exam file.



Search : Search for an existing exam file.



Modify : Modify an existing exam file.



Validate.



Cancel.



Print an exam file.



Permanently delete an exam file and all associated measures.



View previous exam notes.



Attach documents to an exam.

"Exam" Tab

Previous exams



View and print previous exam notes to facilitate patient care. Do not forget to fill in "Title" area and the **exam notes** if you desire to see patient notes history throughout care.


The screenshot shows a window titled "Previous exams" with a print icon at the top left. It contains a form with the following fields: Name (TEST), First name (PODO), Birth date (03/02/1960), Date (04/05/2010), and Title (Check-up). Below this, there are two more exam entries: one with Date 07/03/2008 and Title Pronation, and another with Date 07/03/2008 and Title Supination. Each entry has a large text area for notes.

Attaching documents to an exam



This function allows attaching documents (word, pdf, picture, video) to an exam.

The screenshot shows a window titled "Attached documents" with icons for adding (+), removing (-), editing (pencil), and deleting (trash) documents. It contains a form with the following fields: Name (TEST), First name (PODO), Birth date (03/02/1960), Date (04/05/2010), Title (Check-up), and Filename. Below this, there are two more exam entries: one with Date 07/03/2008 and Title Pronation, and another with Date 07/03/2008 and Title Supination. Each entry has a large text area for notes.

- ✓ Select an exam, by default the displayed exam is selected.
- ✓ Click on  to search on your computer the document to be attached. The software will generate a copy of this document in the directory *c:\win-pod\Documents\Attached* in order to group all attached documents.

"Exam" Tab

Attaching documents to an exam

The following icons will allow you to manage attached documents :



Undo a document attachment. This function does not erase the document from the computer.



Rename the document



Erase a document from the computer.

The software also allows you to automatically attach pdf prints to the current exam, when the arrow is located on the print icon : , or attach images when you right click on the footprint (menu "Attach to the exam").

Creating a new exam

✓ For an existing patient, click on .

✓ Complete the different sections.

✓ Validate by clicking on .

Searching for an existing exam

✓ Click on .

✓ Click on or double-click directly on the exam.

Note : You can only modify measurements in an existing exam on the date of the exam. Later on, you have to create a new exam file.


Modifying an exam file

✓ Click on .

✓ Modify the different sections you selected.

✓ Click on to validate changes.

"Acquisition" Tab

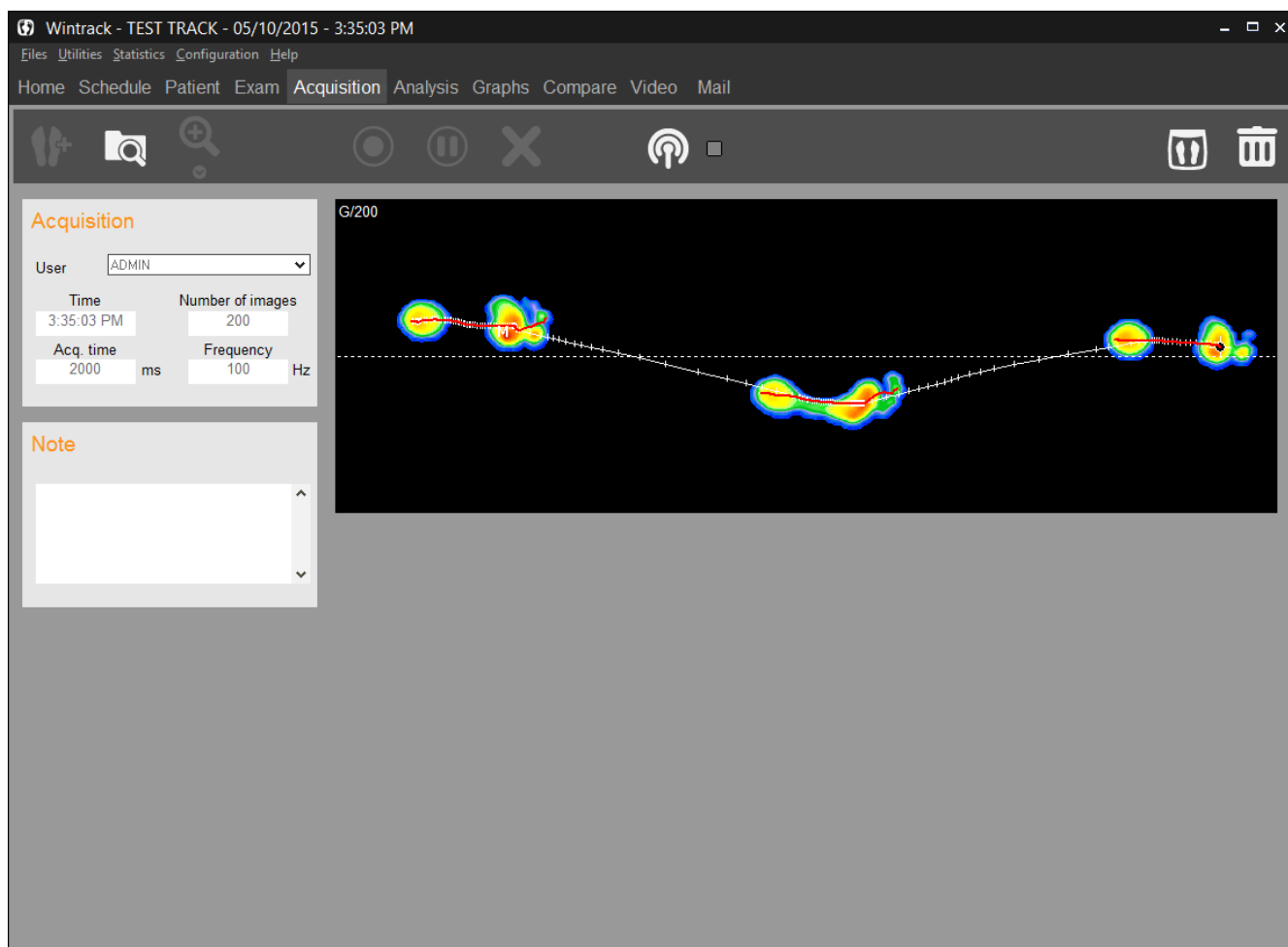

"Acquisition" Tab

A measurement consists of the recording (**acquisition**) of data in three modes (Static, Dynamic (Gait), and Postural).

For each acquisition, all the data are **automatically saved** on the hard disc.

A measurement is always related to a particular exam.

In the course of an exam, you can perform as many measurements as you want (for example, you can carry out 3 static acquisitions and 1 dynamic acquisition).

Note : The comments entered during the creation of an exam file are included on the analysis screens, comparison studies and prints (under the title).

Tool Bar



New : Creating a new measurement file.



Search : Searching for previous recordings.



Modify : Modify an existing measurements file.



Validate.



Cancel.



Bluetooth connection to use the Medicapteurs Remote.



Permanently delete a measurement.



Plate Connection Test: View “real time” images of pressure mapping. This function is very useful to verify proper connection of system and to perform a quick visualization of foot mapping without saving the data.

Warning: This function does not allow acquisition.

"Acquisition" Tab




Creating a measurement (Acquisition)

You can choose 3 types of acquisition:

- **Static**: To record the bipedal cartography of the plantar pressures.
- **Dynamic**: To record the foot progression on the platform through the gait cycle. A high acquisition frequency (up to 150 images per second) allows an accurate analysis of the evolution of plantar support.
- **Postural**: Posturology is in a way "the dynamic of static upright posture ». Instead of recording the patient's plantar supports at a particular time to obtain a single (static) image, recordings of the patient's bipedal support are performed during an established time period in order to analyze the variations of plantar support and the displacement of the foot pressure centers.

It is used for evaluating postural instability or analyzing plantar supports during movements for specific activities (sports, rehab, etc.).

To create a new measure, you can :

- Click on the  icon, then choose the type of acquisition.
- Use the remote you have installed on your mobile device (for more information on the remote's installation and setup, please refer to its users guide).
 - Click on  to open the Bluetooth link from your software. The square indicator next to this icon lits green when the connection is made.
 - Tap on  to open the Bluetooth link from your mobile device. The indicator on this button lits when the connection is made.
 - Then select, validate, or cancel acquisitions from your mobile device.
Comment : The setting windows for dynamic or posturological measures won't appear when you use the remote ; the latest software settings will be used.



"Acquisition" Tab


Static Acquisition


✓ Click on  in the "Type of Measurements" window.

✓ Position the patient anywhere on the platform; the software captures a 50 cm wide area starting from the 1st left active captor along a 50 cm length.

On the screen, you will see a real time image of the cartography of the plantar pressures and the **Center of Pressure (displayed as ⊕)**

You can type comments related to observations in the « Measurements Notes » section.


✓ Start acquisition by clicking on .

When the "Acquisition" key turns gray, the acquisition is completed and the patient may step down. Data is recorded on the computer disc. You can delete by clicking on .


You may now either start a new acquisition or go on to another operation.

"Acquisition" Tab

Basic Dynamic Acquisition (one gait cycle)

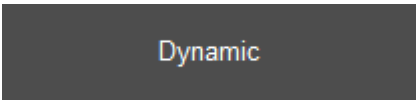- ✓ Click on  in the "Type of Measure" window.
- ✓ The **Acquisition Parameters** screen appears.

Instruct the patient to step down from platform.

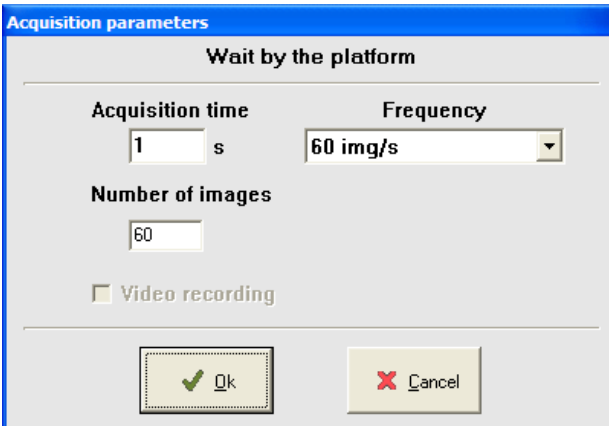


The image shows a software window titled "Acquisition parameters". Inside, there is a sub-header "Wait by the platform". Below this, there are three input fields: "Acquisition time" with a text box containing "1" and a unit "s", "Frequency" with a dropdown menu showing "60 img/s", and "Number of images" with a text box containing "60". There is also a checkbox labeled "Video recording" which is currently unchecked. At the bottom, there are two buttons: "Ok" with a green checkmark icon and "Cancel" with a red X icon.

Acquisition Parameters

Time (in seconds)

Time required by the patient to complete an entire step on the platform. Acquisition time is set by default at 2 seconds. If the patient's gait is extremely slow due to difficulty in moving, this parameter may be changed. The "normal timing" for a full step progression is between 700 and 900 ms.

Frequency

Click on  to select the acquisition frequency. This selection determines the number of images recorded per second (up to 150).


"Acquisition" Tab

Basic Dynamic Acquisition (one gait cycle)

- ✓ Once you have selected the acquisition parameters, click on « OK ».

The following screen appears:

Note : recording starts as soon as contact is made with the platform !

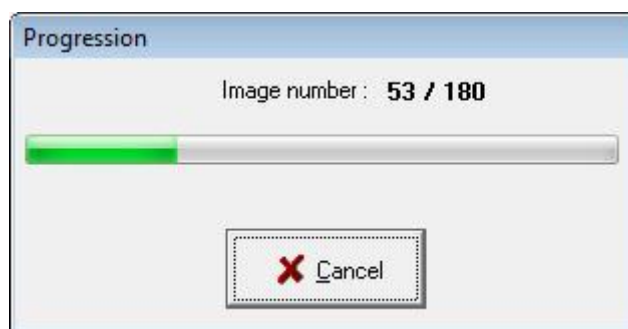

The patient has a 20 seconds period to start walking on the platform (automatic acquisition starts at the first heel strike).

The patient can go over the platform in any direction; the measurement is always displayed from left to right on the screen.


Based on the computer performance and the duration of the recording, the software may require few seconds before displaying the recording.

You may now either start a new acquisition or go on to another operation.

"Acquisition" Tab

Multiple Dynamic Acquisitions (Multiple Steps)

Multiple dynamic

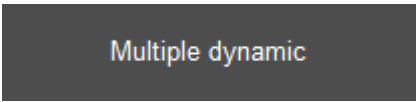
- ✓ Click on  in the "Type of Measurements" window.
- ✓ Select your acquisition parameters (see page 33) and click on « OK ».
- ✓ When the screen appears (see below), the 6 icons "Acquisition" are empty. You can now start acquisitions.

Description

The Multi Steps Option allows acquiring up to 6 dynamic acquisitions.

From these multiple acquisitions, calculations and graphs can be displayed for each foot to describe the evolution of **force**, **maximum pressures**, **average pressures**, **surface areas** during the gait cycle as well as averages throughout the entire process.

The software automatically selects the acquisition which is the closest to the average, allowing the user to visualize the most significant one.

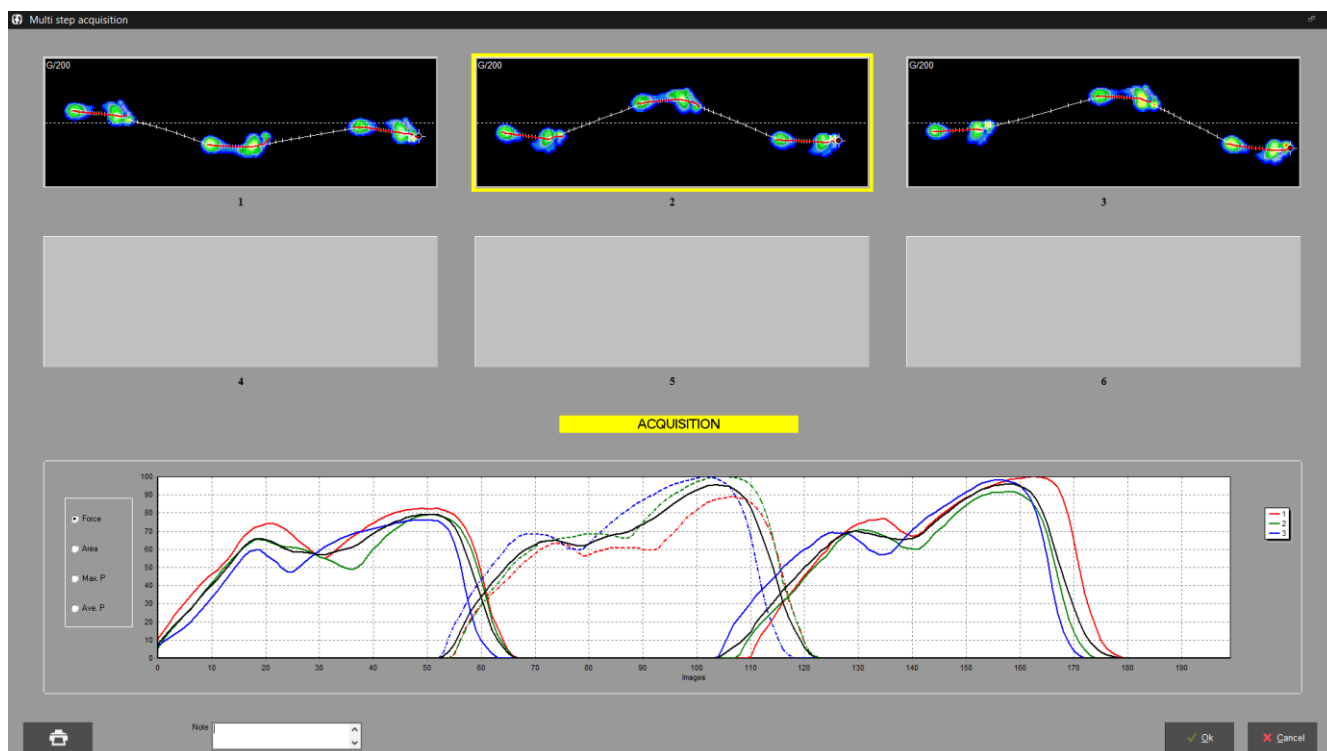



"Acquisition" Tab

Multiple Dynamic Acquisitions (Multiple Steps)

Recording

- ✓ Walk on the platform (up to 6 acquisitions) in which ever direction you prefer always striking the gray portion of the Win- Track platform.
- ✓ After verification and proper selection of images you want to keep (you can **zoom on each image** by double-clicking on it), click on one of the parameters (we recommend "**Force**").

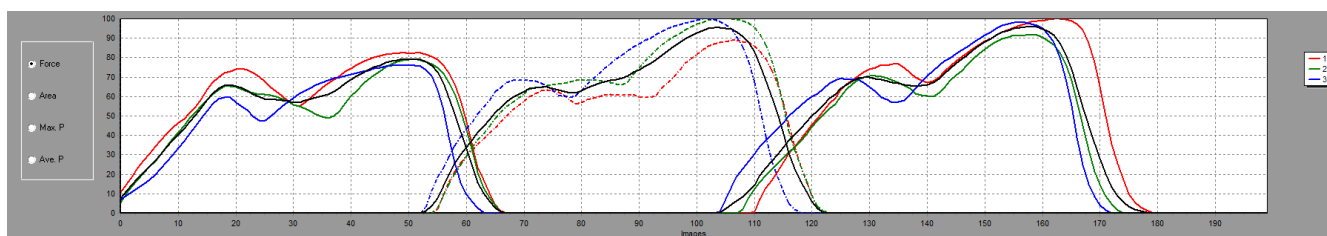


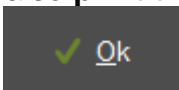
- ✓ The software automatically selects the acquisition which is the closest to the average (the yellow box).
- ✓ If you choose to modify this selection, you can select another image by clicking on it and moving the yellow box.
- ✓ You can also **print the screen** to keep a record of the images and graphs.
- ✓ Click on . A new window appear to allow you to save all the acquisitions you want in the patient file.

You may now either start a new acquisition or go on to another operation.

"Acquisition" Tab

Postural Acquisition

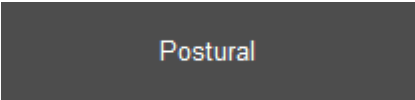- ✓ Click on  in the "Type of Measure" window.
- ✓ The Acquisition Parameters screen appears :

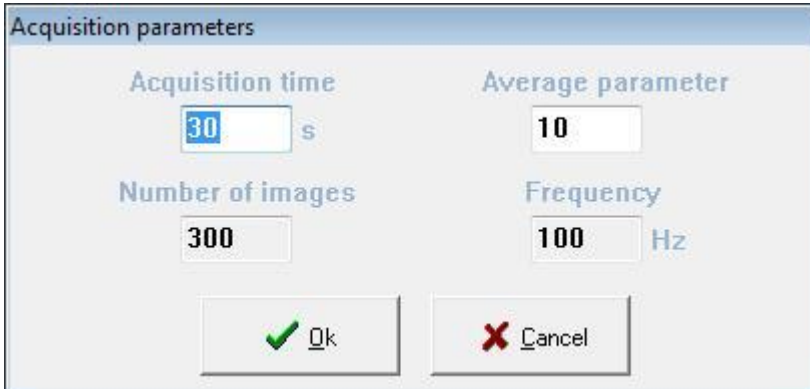

The dialog box titled "Acquisition parameters" contains four input fields arranged in a 2x2 grid. The top-left field is labeled "Acquisition time" and contains the value "30" followed by a unit "s". The top-right field is labeled "Average parameter" and contains the value "10". The bottom-left field is labeled "Number of images" and contains the value "300". The bottom-right field is labeled "Frequency" and contains the value "100" followed by a unit "Hz". At the bottom of the dialog are two buttons: "Ok" with a green checkmark icon and "Cancel" with a red X icon.

Acquisition parameters

Time (in seconds)

The duration of the postural analysis recording is 30 seconds by default.

Average Parameters

Postural acquisition is always performed with a frequency of 100 Hz (100 images per second). The average parameter permits reducing the volume of data (limited to 2400) to be saved on the hard disk to avoid saturation:

$$\text{Number of images saved} = \frac{\text{Duration} \times 100}{\text{Average Parameter}}$$

Example: Duration of acquisition: 30 seconds

Average Parameter	Acquired Images	Saved Images
100	300 x 100 = 3000	30
10	300 x 100 = 3000	300
1	300 x 100 = 3000	3000 = over limit

"Acquisition" Tab

Postural Acquisition

Acquisition Parameters

Number of images (non modifiable information)

The number of images saved during a recording.

$$\text{Number of images} = \frac{\text{Duration} \times 100}{\text{Average Parameter}}$$

Frequency (non modifiable information)


Number of images recorded per second. This parameter cannot be modified. It is set at 100Hz.

- ✓ When parameters are selected, click on "OK".

Position the patient on the platform. The patient has to face the box; **rotation cannot be performed** on 1m wide measurements.

The software registers a 50cm or 1m area starting from the first active left captor based on foot positioning on the platform, along a 50cm height.


- ✓ Click on . Acquisition starts :

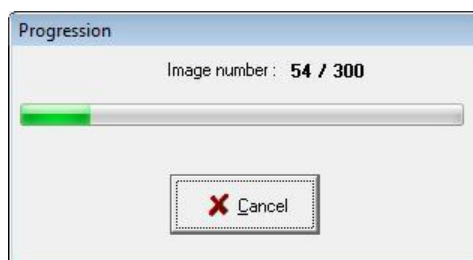

Warning : Acquisitions use all computer resources and therefore other computer functions cannot be performed during acquisitions ! Particularly live applications such as: antivirus, scan modem, downloading, skype...and with applications requiring large memory use such as: large word files, keyboard, etc.

When acquisition is completed, you may now either start a new acquisition or go on to another operation.

Detailed Static Analysis

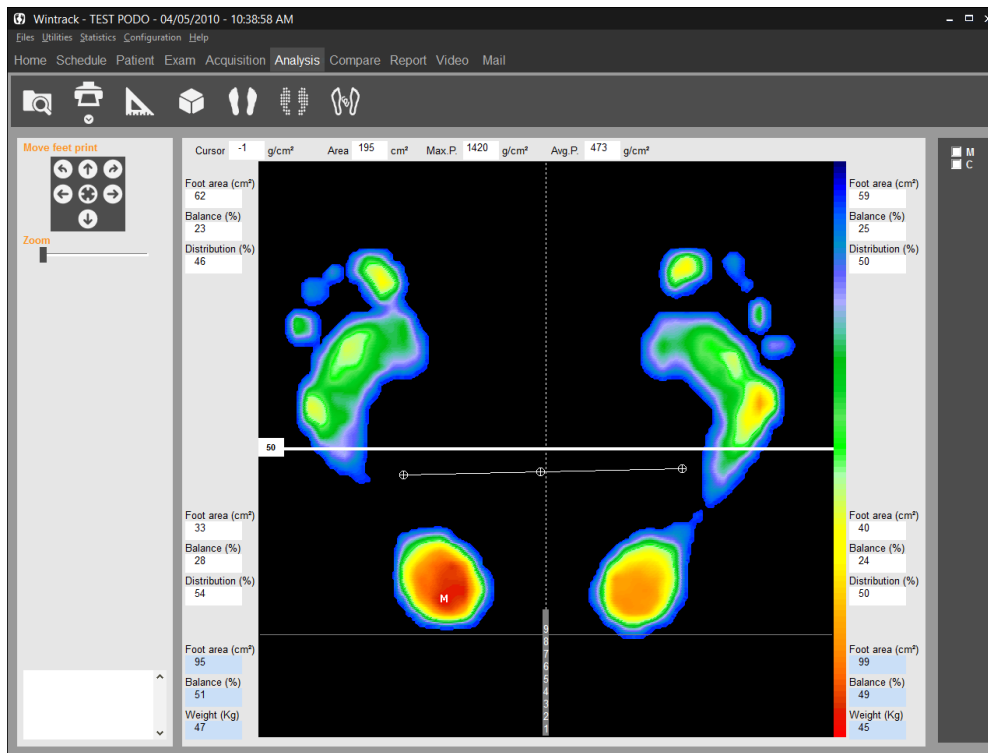

Tool Bar



Search : Opens the patient database to review a measurement.



Print. The arrow gives access to "Scale 1 Printing" menu (see page 79) and " Attaching documents to an exam" (see page 27).



Foot Print Measurements.



3D view.



Standard view.



Isopressure view.



Center of Pressure visualization.



Foot print centered and calculation of « Bessou Line Analysis » according to the center of pressure.



Foot print centered and calculation of « Bessou Line Analysis » according to the point of maximum pressure.

Detailed Static Analysis

Foot Print Analysis

Centering the Foot Print



These keys are used for framing or rotating the foot print for easier review when your patient is not correctly positioned during acquisition or to modify the forefoot/heel limits used for calculations by sectors.

Important Note : When you re-frame a print, the initial print file is automatically replaced by a re-framed print file, on the hard disk (if you then open the print file, you will be able to see the re-framed print).

Cartography

The following information is displayed on the **cartography**:



: Point of maximum pressure (the value is displayed in Max P.

⊕ : These 3 marks depict the COP (Left foot, Right foot, and Center). The center ⊕ is the **Center of Pressure**.

Calculation on both feet

Cursor	1203	g/cm ²	Area	195	cm ²	Max.P.	1420	g/cm ²	Avg.P.	473	g/cm ²
--------	------	-------------------	------	-----	-----------------	--------	------	-------------------	--------	-----	-------------------

- **Cursor**: The pressure measured at the point where the cursor is located(g/cm²). Use the mouse to navigate this tool.
- **Area**: Surface Area covered by both feet (total support surface) in cm².
- **Max P.**: The maximum pressure point measured.
- **Avg P** : Average pressure calculated over both feet.

Detailed Static Analysis

Foot Print Analysis

Calculation per quadrant

Several calculations are performed in each of the 4 image sectors (quadrant).

The following parameters are available :

For each quadrant (in white)

- **Area** : Support surface in the quadrant (cm²).
- **Thrust** : Percentage of Thrust in a quadrant, as a proportion of the total pressure through **both feet**.
- **Distribution** : Percentage of support in the front and back halves of the foot.

For each foot (in blue)


- **Area** : Support surface of the foot(cm²).
- **Thrust** ; total thrust of the foot.
- **Weight** applied on the foot.

"Standard" View



The plantar pressures mapping is represented with a palette of different colors from blue (**low pressure**) to red (**high pressure**).

Note : The image resolution, screen display and the colors can be changed in the Configuration Menu : Configuration/Parameters/ Colors (see page 17).

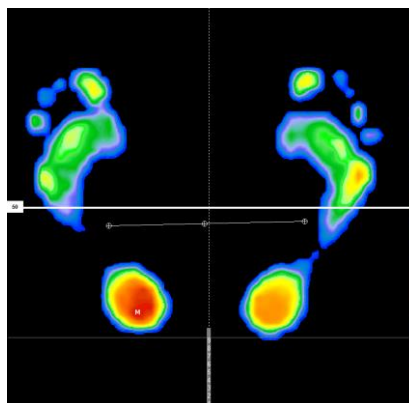

Detailed Static Analysis


"Isopressure" View




In the standard view, the different levels of plantar pressures are represented with a continuous palette of colors from blue to red.

In the isopressure view, there are only 3 colors available (red, green and blue) corresponding to 3 pressure ranges:


- Blue : Low pressures.
- Green : Intermediate pressures.
- Red : High pressures

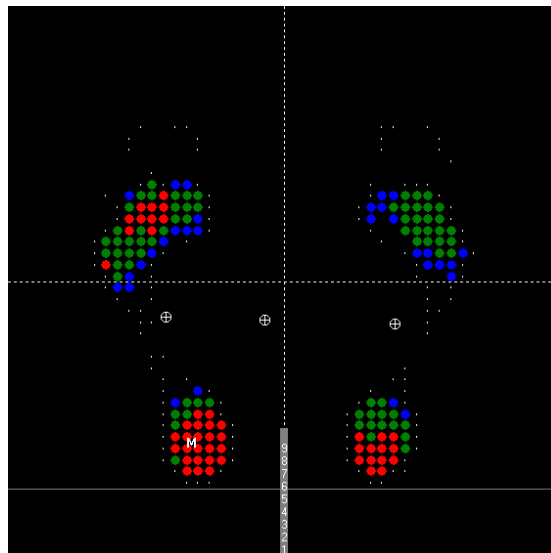

The pressure level which determines each color can be adjusted by the user by **increasing or decreasing** the pressure level corresponding to a color by clicking on  (see example below).

The value which is on the right of the color square represents the pressure level (expressed as a percentage of the maximum pressure point "M").

Example :

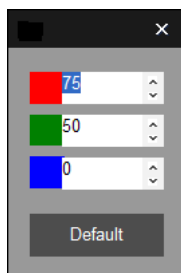


Red represents areas of pressure **above 75%** of the max pressure.
Green represents areas of pressure **between 50% and 75%** of the max. pressure.
Blue represents area of pressure **between 0% and 50%** of the max. pressure.

Detailed Static Analysis

“Center of Pressure” View



This mode is used for visualizing the foot shape, the centers of pressure (\oplus) and the max. pressure point (M). It is often used in dynamic analysis to display the “Gait Line”.

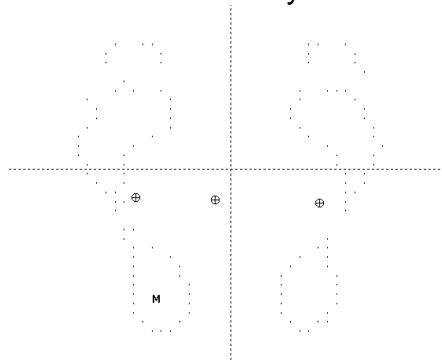




"3D" View



The cartography of pressures is visualized in 3 dimensions.

The object of this image is not to permit an accurate analysis of support, but rather to produce an easily understandable and highly visible image for the patient.

The picture can be moved and rotated by clicking on these arrows   or by moving along the “Depth” cursor.

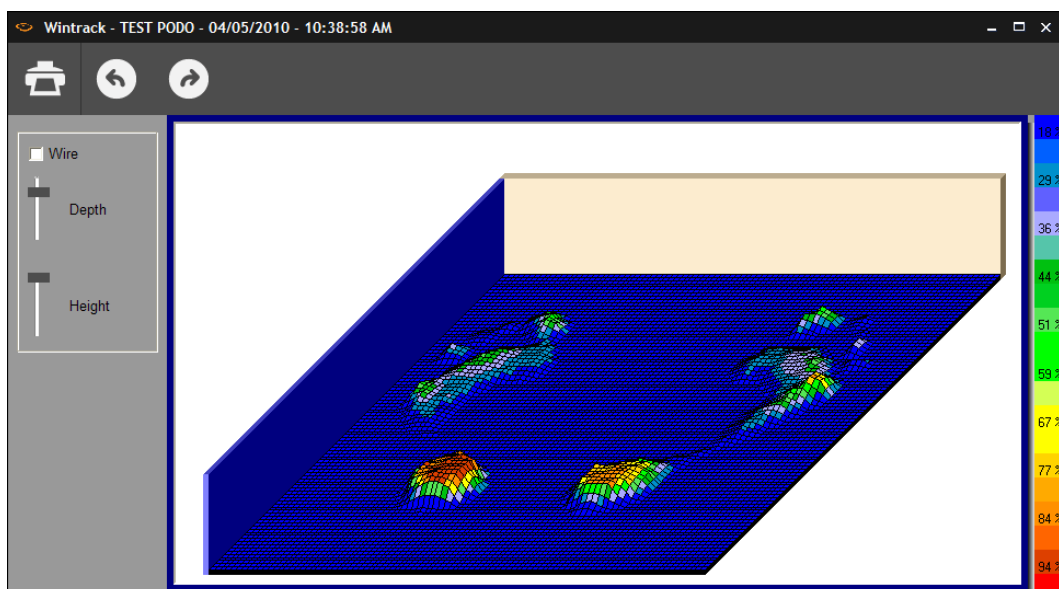


Detailed Static Analysis

"Measurements" Option



Measurements can be carried out and **comments** can be added on the cartographies. It will appear on the screen and printing but **it is not recorded**.

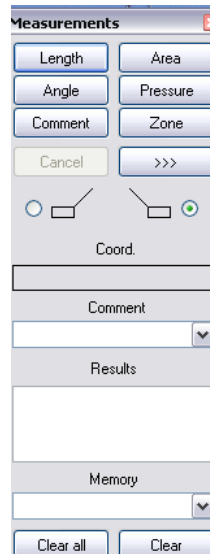

Click on the option to select:

- **Length** : Measuring one or more distance on the cartography in cm.
Point and click (once) to define the first point of the segment to be measured. Move Mouse, point at a second point, then click.
- **Angle** : Measuring an angle on the cartography. Point and click the 3 points forming the 2 generating lines of the angle to be measured.
- **Area** : Click once, and then move mouse to form an ellipse. Click a second time to display the surface area within the ellipse.



Detailed Static Analysis

"Measurements" Option

- **Zone**: Maximum Pressure, Average Pressure, Surface Area, Timing calculations in a selected area (following the captors' distribution).
To open a zone, click on the upper left corner of the selected zone, keep the pressure on the mouse and move it towards the right lower corner. When you release the mouse, the measurements values will appear.

Pressure: Click on point on the cartography to display the pressure at that particular point. You can click on the box with pressure value to move it and then double click to leave it at the position you selected.

- **>>>**: Insert an **icon** directly on the foot print.
- **Comment**: Used for inserting comments on the cartography (will appear when printing)

- ✓ Click on 
- ✓ Click on the beginning of the line 
- ✓ Type comment on the selected foot print area.
- ✓ Click on point to insert the comment.
- ✓ Move mouse to position comment box.
- ✓ Click a second time to have comment fixed.



- **Results**: Displays the results of the last measurement performed (length, angle or surface).
- **Coord.**: Displays the Cartesian coordinates (x,y) of the position of the mouse cursor.
- **Clear**: Deletes the **last** performed measurements.
- **Clear all**: Deletes **all** the performed measurements.



Detailed Static Analysis

"Lines of Bessou"


Presentation

Professor Paul Bessou, affiliated with the Podiatry School of Toulouse, France presented new description methods to clearly define static foot prints and particularly compare to measurements with or without treatment plans (for example with or without orthotics).

In the "Detailed Analysis » tab section, you can click on ,  the following lines and calculations appear :

- Internal and external foot tangents.
- The lines passing through the fore foot and heel centers of pressure (.
- The lines passing through the maximum pressure points in the fore foot and heel (.
- The line connecting the back of the heels (which should be aligned)

A reference was given for each of these parameters :

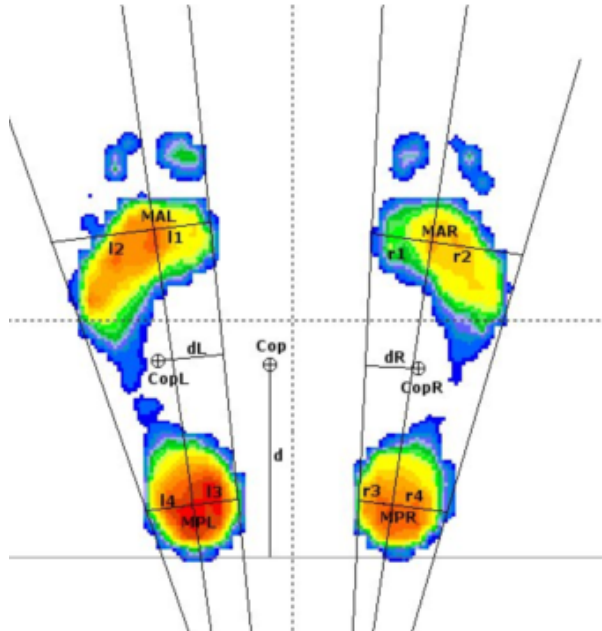

MAL : Max Anterior Left
MPL : Max Posterior Left
MAR : Max Anterior Right
MPR : Max Posterior Right

l1 : MAL – internal left tangent distance
l2 : MAL – external left tangent distance
l3 : MPL – internal left tangent distance
l4 : MPL – external left tangent distance

r1 : MAR – internal right tangent distance
r2 : MAR – external right tangent distance
r3 : MPR – internal right tangent distance
r4 : MPR – internal right tangent distance

Cop : Center of pressure
CopL : Center of pressure Left
CopR : Center of pressure Right

d : Cop and heels line distance
dL : CopL and internal tangent distance
dR : CopR and internal tangent distance
Angle : angle between internal tangents

Find out how to use the references on the next page.

Detailed Static Analysis

"Lines of Bessou"

Use

From the lines automatically displayed, the software calculates multiple measurements of distances to define the forefoot degrees of **rotation** (**l1** and **l2** for the left foot, **r1** and **r2** for the right foot) or for the heel (**l3** and **l4**, **r3** and **r4**).

The calculation of the **distance "d"** between the line posterior to the heels and the global center of pressure provide information on the antero posterior positioning of the patient: the greater the distance the greater the anterior positioning of the patient.

Finally, between the distances **dL** and **dR** between the lines passing through the global center of pressure of each foot and the internal tangents quantifies the global rotation of each foot.

The explanation of these parameters is extensive however they are really **easy to use** and it allows quantifying the effects of treatments on patients:

- ✓ Perform to successive recordings that you want to compare (typically with and without orthotics).
- ✓ Display the « lines of Bessou » according to M or C.
- ✓ Select the parameters you want to study and compare them with the results of the 2nd recording.

For example for the correction of excessive forefoot external rotation, compare the distances r1/r2 and l1/l2.

Similarly, study the evaluation of the parameter "d" to verify the effect of the treatment on the patient's antero-posterior position.

In 2006, the National Association for Research In Podiatry (ANREP) launched a study with MEDICAPTEURS and the French post graduate institutes on the effect of the supination corner on plantar pressures.

The WIN-POD system was selected for this particular study and the software was modified to include the « lines of Bessou » used for quantifying plantar pressure changes.

Detailed Dynamic Analysis



Detailed Dynamic Analysis

Tool Bar



Search : Access to patient file to load measurement.



Print. The arrow gives access to "Scale 1 Printing" menu (see page 79) and " Attaching documents to an exam" (see page 27).



Foot print measurements.



3D view.



Standard view.



Isopressure view.



Foot Outline view, Gait line and Max line.



Contact duration.




Impulse view (Time x Pressure).



Start or stop the step progress.



and  In mode pause, it permits to see a forward or backward image by image replay.



Displays the first image of the step progress.



Displays the mean results of all the progress images(global view).



Left Foot Display.



Right Foot Display.



2 Feet Display.



Mosaic : display all the progress images on one screen.



Displays the curve of displacement of the Max. support(Maxline).



Displays the curve of displacement of the center of thrust (Gaitline).



Displays the three phases of the step.


Detailed Dynamic Analysis

Foot Print Analysis

Tools of visualization

- **Zoom** : this pointer allows to enlarge a particular measurement.

When Zoom is activated, a little box located in the upper left corner of the foot print displays the entire recording and inside a yellow rectangle emphasizes the enlarged selected area. This rectangle can be moved with the mouse (by clicking and holding on the mouse) so it can be moved easily along the recording.

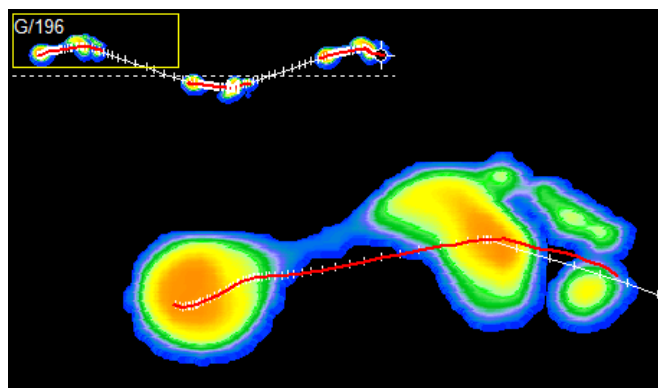

Return the Zoom pointer to 0, or click on  in the toolbox to cancel the zoom.

- **Speed of progression** : by moving the cursor, the speed of image display can be changed.

On the cartography

On the cartography, the following information is available :


417 ms : Time elapsed since the beginning of acquisition. Expressed in milliseconds (ms).

If the  **M** and  **C** are not selected:



: Point of maximum pressure.

⊕ : Centre of de thrust calculated on the displayed image.

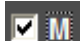
If  **C** and/or  **M** is/are selected :

Visualization of the **Gaitline** (curve representing the evolution of the thrust center) and/or the curve representing the evolution of maximum point of support (**Maxline**).

Detailed Dynamic Analysis

Foot Print Analysis

Calculations on displayed image


Cursor	2281	g/cm ²
Cadence	81.4	Step/mm
		
Area	141	114 138
Avg P.	1097	1224 1192
Max P.	2281	2880 2677

- **Cursor** : The pressure value of the support point which is pointed at with the mouse arrow is displayed in this box (g/cm²).
- **Cadence** : Cadence foot for the displayed image.

For each foot

- **Area** : Support surface of the foot (cm²).
- **Max P.** : The maximum pressure point measured.
- **Avg P.** : Average pressure calculated over both feet.

The phases of the gait cycle

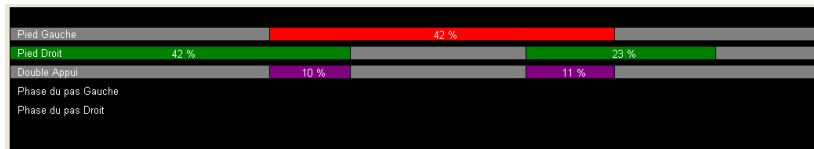

The first line of information displays the duration of the first foot strike in comparison to the entire duration of the walk through the platform.

La second line of information displays the duration of the foot strike of the other foot in comparison to the entire duration of the walk through the platform.

The third line of information displays the phases when both feet are on the platform at the same time.

Force graphs



These graphs represent the impact force for each foot using the colors assigned for each foot in the « phases of the gait cycle » section as well as when both feet are on the platform at the same time. The thicker graph displays the sum total of the forces.

Detailed Dynamic Analysis

Foot Print Analysis

Space-Time Calculation



This key allows displaying the Space-Time calculation window. The results can be




printed by clicking on . Results are printed along with the foot pressure map as displayed on the analysis tab.

- **Step duration and length** : Timing and distance between the heel strike of one foot and the heel strike of the other foot.
- **Gait cycle duration and length** : Timing and distance between the heel strike of one foot and the next heel strike of the same foot.
- **Single stance duration** : Duration of the portion of the gait cycle when only one foot is in contact with the ground (around 40% of the cycle).
- **Double stance duration** : Timing when both feet are in contact with the ground (around 10% of the cycle).
- **Swing duration** : Time during which one foot is off the ground (around 40% of the cycle).

Note : the duration of a single stance is equivalent to the duration of the swing on the opposite foot.

- **Stride duration** : Time between the 1st contact on the ground and the last contact on the ground of the 2nd step for the same foot.

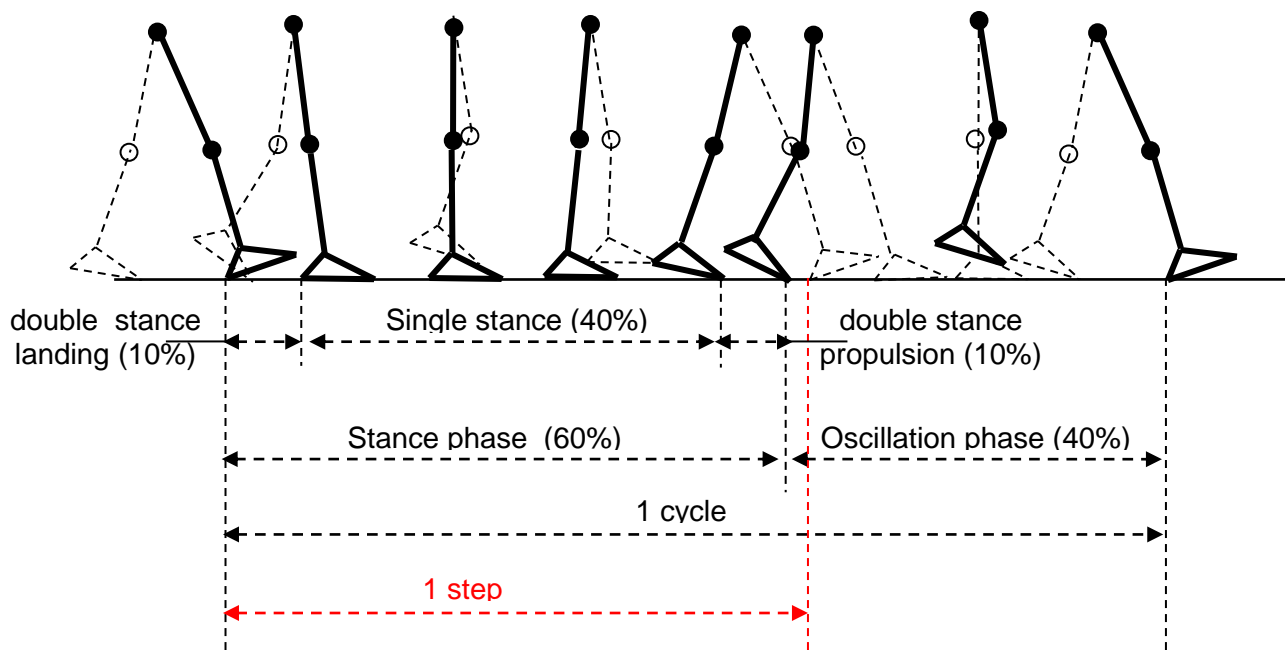

Detailed Dynamic Analysis

Multiple Views



Standard view (See page 41).



Isopressure view (See page 42).



Center of Pressure view (See page 43).



« Duration of contact » view.

This image displays the time length of foot support in each foot area during the step process : the darker the blue color, the longer the foot support lasts.



"Time x Pressure" view (or "Impulse").

In dynamic analysis, hyper pressures and duration of contact can be studied, however the most interesting parameter is the **"Time x Pressure"** which defines **"Quantity of Charge"** applied for each foot area and allows to outline potential "risk areas" (particularly as a preventive measure for the "diabetic foot").


The cartography **"Time x Pressure"** can be easily analyzed : the darker the red color, the greater the charge quantity.




3D view (see page 43).




Measurements options (see page 44).

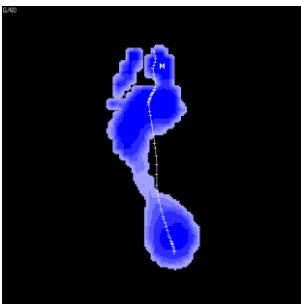


Contact Duration

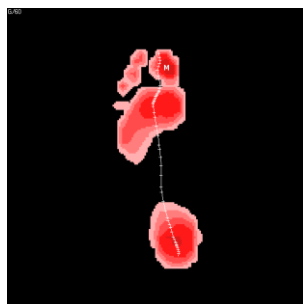

Impulse

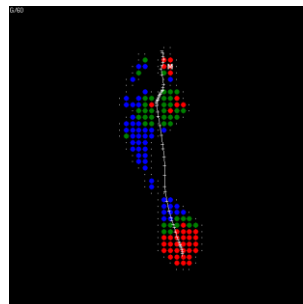


Isopressure

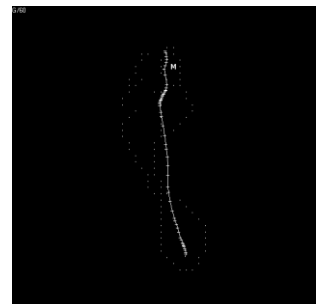

Center of pressure

Graphs – Dynamic

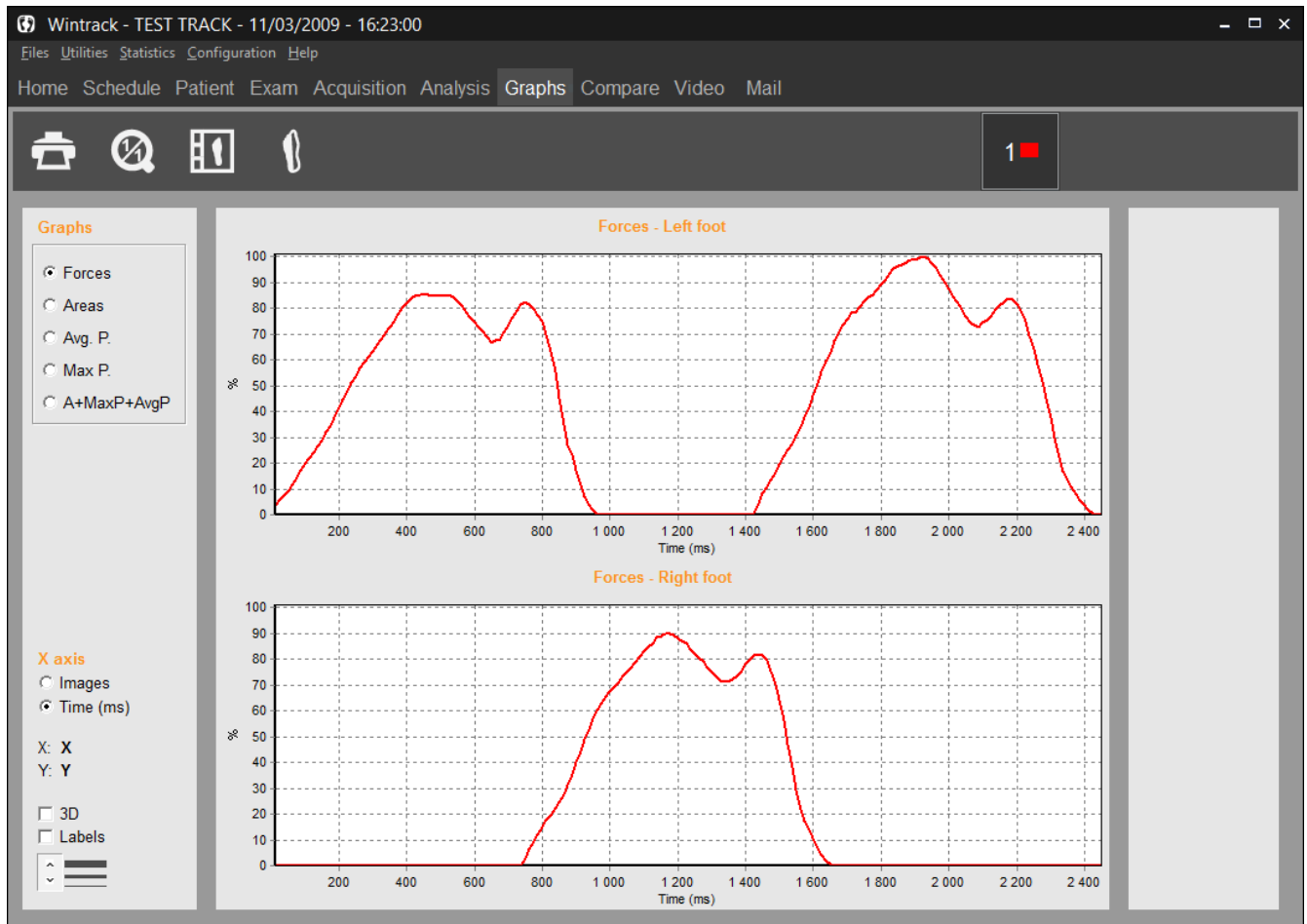

Graphs – Dynamic

A certain number of parameters are calculated for each image of the dynamic progress (Surface, Max. Pressure, Aver. Pressure, etc).

The evolution of these parameters throughout the step cycle can be visualized by selecting the "Graphs" tab.

Tool Bar



1 : 1 scale. You can zoom in on a graph by opening a rectangle with the mouse on the selected portion of the graph to be enlarged. This button allows bringing back the original image.



Printing graphs.



Synthesis of the feet.



Medio-Lateral Analysis.

Graphs – Dynamic

Selecting a graph


Click on the curve you want to be displayed :

- **Forces** : Evolution of the force during step progression (%).
- **Area** : Evolution of the support surface during step progression (cm²).
- **Avg. P.** : Evolution of the average pressure calculated for each image (kg/cm²).
- **Max P.** : Evolution of the value of the « Max.pressure» through the images of the step progression (kg/cm²).
- **Area + Max.P. + Avg.P.** : Superposition on the same screen of the area, average pressure and maximum pressure graphs.

Graphic options

- **3D** : 3D Visualization of the graphs.
- **Labels** : Displays the values of each image (to be used in conjunction with the zoom function to facilitate reading the values).



-  : Selecting the thickness of the graph lines.
- **X, Y** : Cartesian coordinates for each point you select on the graph.

Graphs – Dynamic


Feet synthesis



This function allows to on the same screen the Area, Avg. P. and Max. P graphs with the step progression view. This is a great page to print to provide **a report of the dynamic analysis on only one page**.



These icons, located above each foot pressure map, allow to select the feet to be compared.

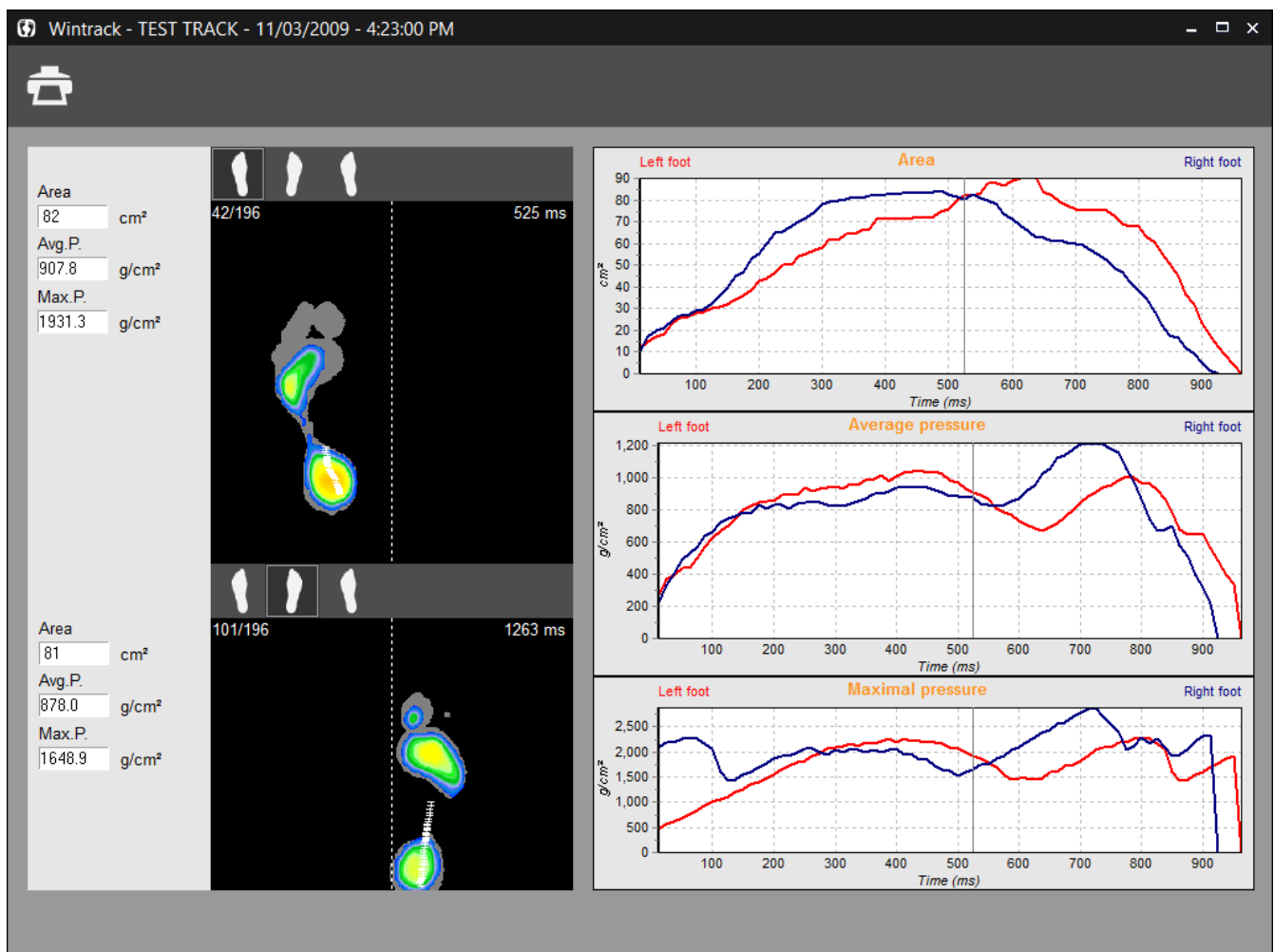


Note : You have the option to visualize the foot pressure mapping (on the left side of the screen) at any time of the step process. Just click on the graph, hold and move the mouse from left to right.



To see the global image, move the cursor to the right side of the graphs.




Graphs – Dynamic

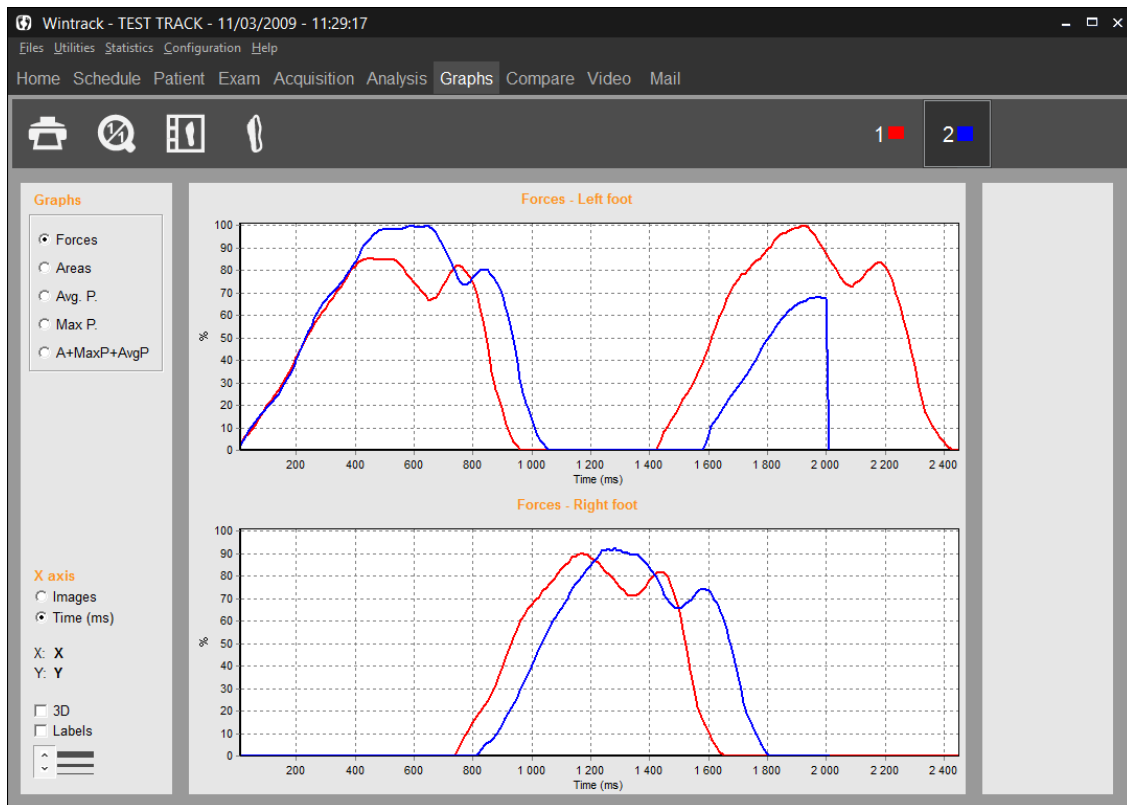Comparing 2 measurements

When several dynamic recordings are loaded in the "Compare" tab section (up to 4), you can superimpose the graphs associated with each recording in the "Graphs" tab :



The icons   allow managing the measurements to be compared.


- By bringing the mouse on these icons, a message is displayed indicating the name, first name, date and time of the recording.
- When a button is selected, the active measurements which are the measurements displayed in the "Detailed Analysis" tab or the measurements displayed when clicking on the icon .
- To stop visualizing a measurement (measurement is still loaded in the software), right click on the corresponding icon. A « cross-like » icon  appears on the screen to replace the previous icon . Follow the same procedure to visualize the measurements again.

Graphs – Dynamic

Medio-Lateral Analysis



This function allows studying the evolution of several parameters in selected medial and lateral areas of the foot.

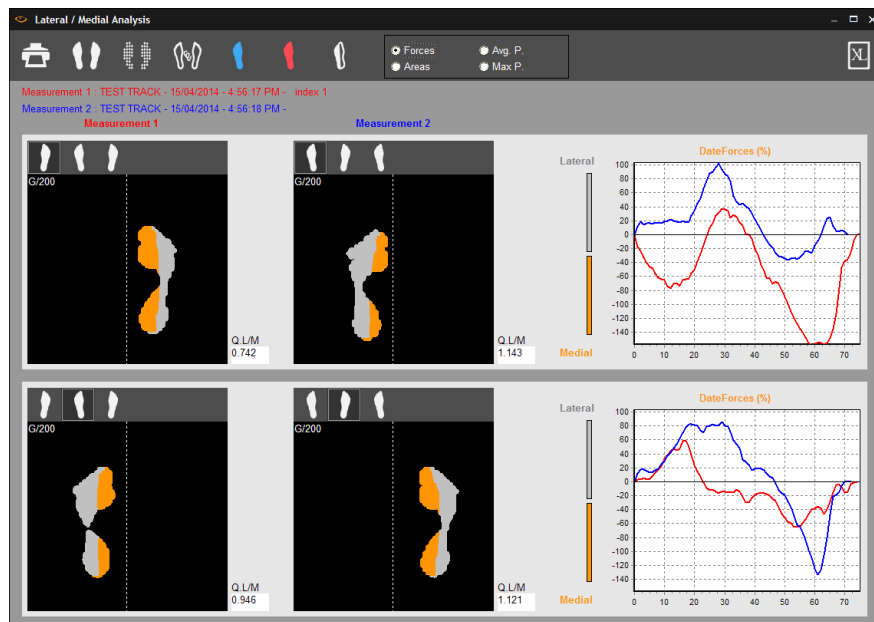

The foot is divided into 2 zones on both sides of the "**Gait line**" (evolution curve for the center of pressure). The **Gray** Color (for the **Lateral**, external side of the foot) and **Orange color** (for the **Medial**, interior side) make it possible to display the zones used for calculations.

The graphs represent the evolution over time of the **difference between the forces on the lateral side and the forces on the medial side of the foot (FL - FM)**. If the lateral value is stronger than the medial value, the curve is positive, and indicates the prevalence of the lateral forces.

These curves are very useful to evaluate and quantify **the torsion of the foot** during the gait cycle as well as the phases of **Heel strike**, **stabilization** and **propulsion** which are difficult to evaluate clinically.


The studies of these functions have been of particular interest to orthopedic surgeons.

L/M index is located on the bottom right corner of each screen displaying the calculation of the relationship between the lateral and medial forces. When the index is equal to "1" it indicates that the medial and lateral forces are equal (which corresponds to a curve passing through the origin of the graph: $FL - FM = 0$).

By moving the mouse cursor on the curves, the particular foot print and L/M index will be displayed for that selected time of the gait cycle.

Note : If 2 dynamic measurements are loaded in "Comparaison", you can compare the medio-lateral analysis of these 2 measurements.

Detailed Postural Analysis

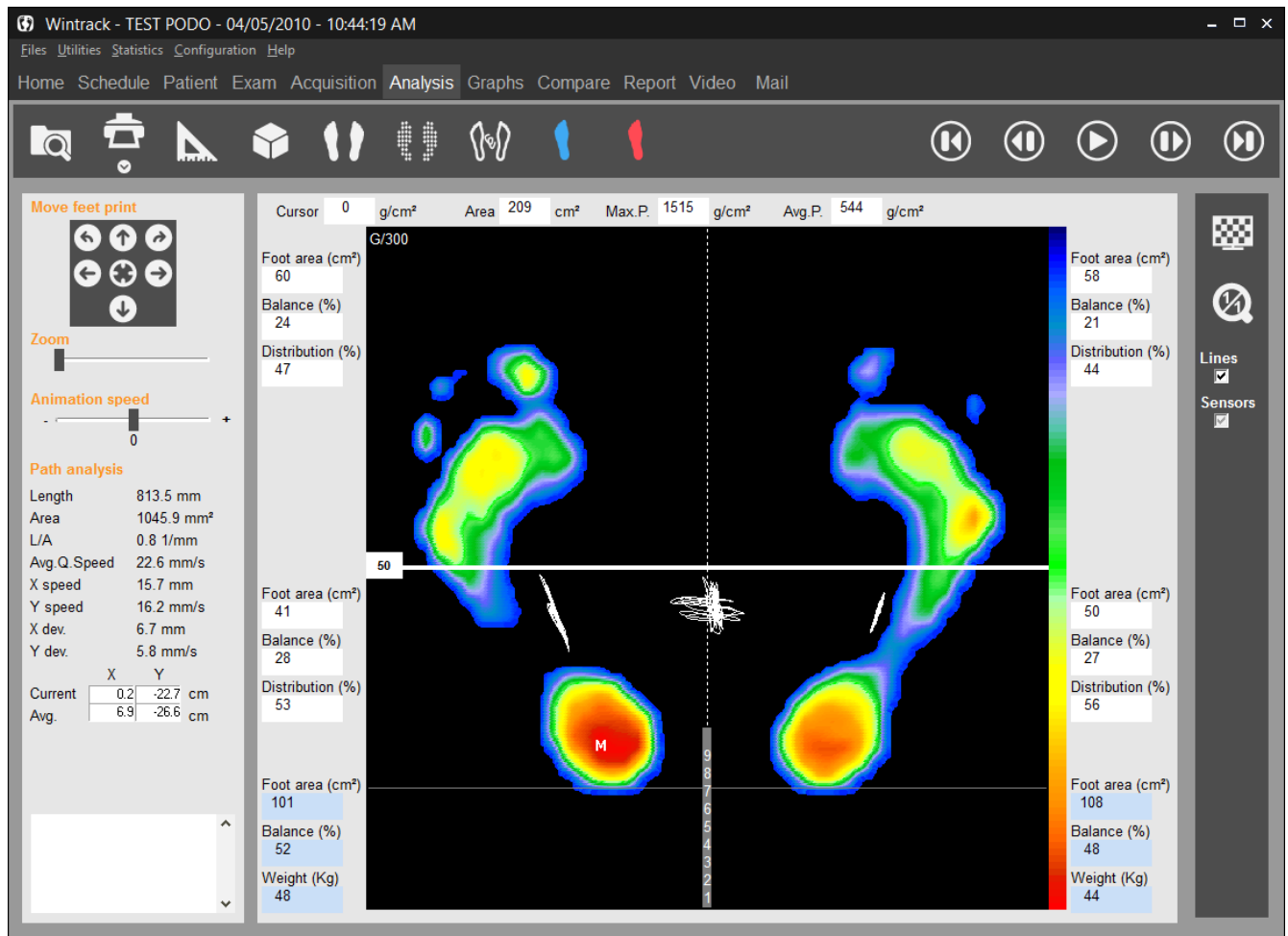

Detailed Postural Analysis

Tool Bar



Search : Access to database to load a measurement.



Print. The arrow gives access to "Scale 1 Printing" menu (see page 79) and "Linked to the exam" (see page 27).



Performing Foot Print Measurements.



3D view.



Standard view.



Isopressure view.



Visualization of the foot print shape and Center of Pressure.



Contact duration.



Impulse view (Time x Pressure).




Return to normal scale 1 : 1.



Starts or stops the step progression.



and  In « pause » mode, it permits visualizing one image after another (forward or backward).



Display the first image of the step.



Display the mean results image of all the images of the step process (global image).



Mosaic : display all the images of the step process.

Detailed Postural Analysis

Foot Print Analysis

Visualization tools

- Moving the foot print :



These keys are used for centering or rotating the foot print in case the patient was not properly positioned during the acquisition or to modify the forefoot/heel limit used for measurement by sectors.

Important Note : when you re-frame a foot print, the original recording in the database is automatically replaced by the re-framed data (if you then open this footprint, you will only see the re-framed version).

- Speed : The display speed of the acquired images can be modified by moving the cursor.
- Lines : Connect or not the thrust centers.
- Shape : in the "Center of Pressure" mode, display or do not display the feet shape.

Cartography

On the **cartography display**, the following information is available:



: Maximal pressure point (the absolute value is displayed in the Max P box).



: These three graphs (oscillograms) display the center of thrust under each foot and the global center of thrust primarily used in postural analysis.



: Time elapse since the beginning of the acquisition in milliseconds (ms).

Detailed Postural Analysis

Foot Print analysis

Calculation on both feet

Cursor	838	g/cm ²	Area	209	cm ²	Max.P.	1515	g/cm ²	Avg.P.	544	g/cm ²
--------	-----	-------------------	------	-----	-----------------	--------	------	-------------------	--------	-----	-------------------

- **Cursor** : In this box is displayed the foot pressure value on the point selected with the mouse (g/cm²).
- **Area** : Total surface area of both feet (cm²).
- **Max. P.** : Point M value : Maximal pressure on both feet (g/cm²).
- **Avg P.** : Average pressure on both feet.

Calculations per sectors

Several calculations are performed in each of the **4 image sectors** (quadrants).
The following parameters are:

For each sector (in white)

- **Area** : total foot print surface area in the sector (cm²).
- **Thrust** : Percentage of thrust in the sector, proportionally to the **2 feet** total thrust.
- **Distribution** : Percentage of support in the anterior sector and the posterior sector for each foot.

For each foot (in blue)

- **Area** : foot print surface area (cm²).
- **Thrust** total foot thrust.
- **Weight** distribution by foot.

Detailed Postural Analysis

Foot print analysis

Calculations related to the trace (curve between the feet)

- **Length** of the trajectory covered by the center of thrust (in all directions).
- **Area** of the closest ellipse to the stabilogram.
- The **Length/surface** ratio provides a “general value” of energy spent by the subject during the recording.
- The **quadratic average speed** parameter represents the vector sum of the antero-posterior and latero-lateral average speeds. This is not an algebraic sum but rather the square root of the sum of squares of the two previously mentioned speeds.
- The **average speed** (expressed in mm/sec) corresponds with the projection on the axis (Lat. in latero-lateral or frontal plan or Ant. in antero-posterior or sagittal plan) of the total displacements of the centre of thrust divided by the duration of the recording. Please note the displacements are measured in absolute value without addressing the direction of the patient's movement. An elevated average speed of oscillations may indicate **postural instability**.
- The **variance or standard deviation** measures the scattering in relationship with the average position of the center of thrust. From a pure statistical point of view, 68% of the oscillations have amplitude between +1 and -1 standard deviation, 95% of the oscillations have amplitude between +2 and -2 standard deviations.
- **Cartesian Coordinates** of the global center of thrust averages: Average and on the displayed image: Current.

Detailed Postural Analysis

The different views



Standard View (See page 41).



Isopressure View (See page 42).



Center of Pressure View (See page 43).

In this view, you can perform one or several **zooms on the oscillograms**.

For zooming function:

- ✓ Click while pressing the key in the top left corner of the area to be zoomed.
- ✓ Move diagonally the mouse toward the bottom right area.
- ✓ Adjust the size of the area to frame the graph or the portion of the graph to be zoomed and then release the mouse key.



To return to the initial view, click on



Contact duration (See page 53).




Impulse View (Time x Pressure) (See page 53).




3D View (See page 43).




Measurements Option (See page 44).

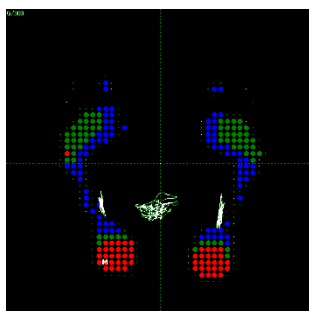


Isopressure

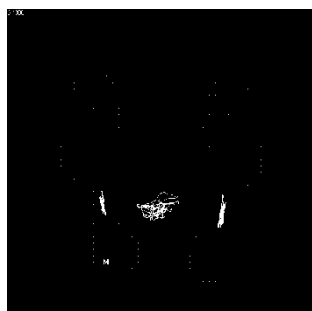

Center of Pressure

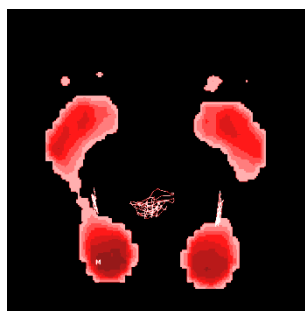

Impulse

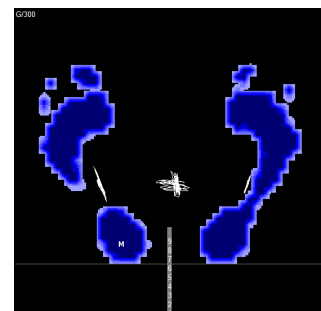


Contact duration

Detailed Postural Analysis

"Mosaic" view



This mode is used for the analysis of major movements or high instability. It allows viewing all the acquisition images simultaneously and looking for the movement phases.

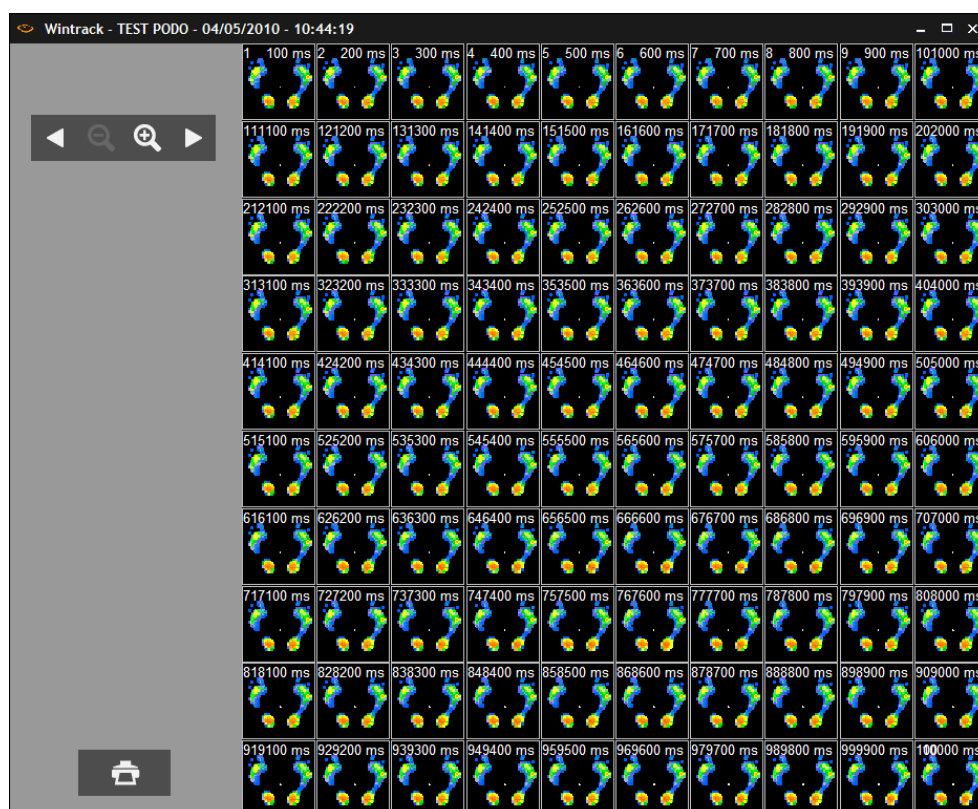

Front Zoom.



Back Zoom.



and



Forward or Backward display.




Print.

Note : When you double-click on one image in the mosaic view, you directly access the detailed analysis of that image.

Graphs - Postural Analysis

Stabilometry analysis is performed through the study of 3 graphs associated with calculations of several physical parameters.

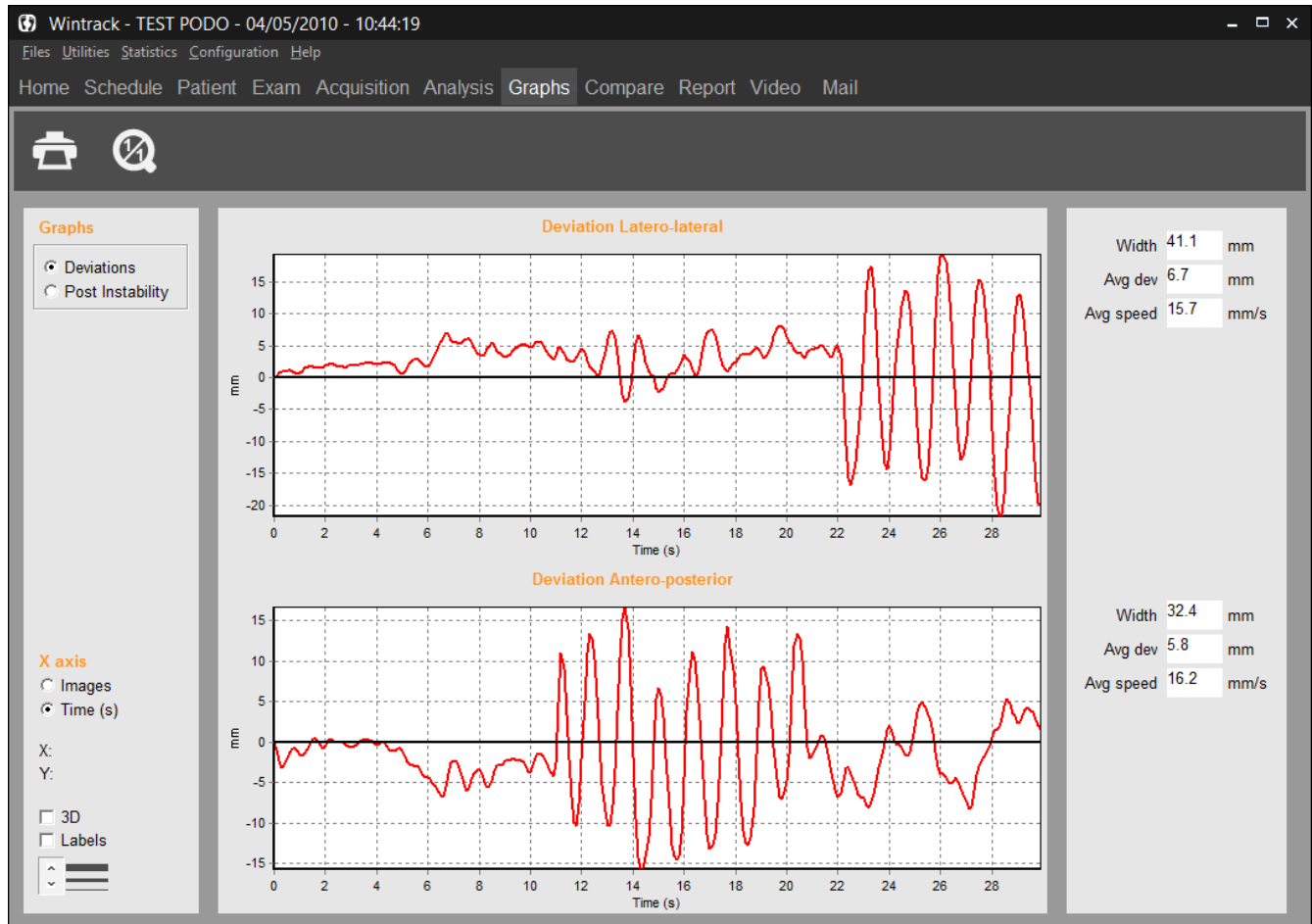


Graph Selection

Click on the graph you want to select:

- **Deviations** : These graphs display the progression of the center of thrust in the antero-posterior and latero-lateral axes during the recording. The vertical values cover a range between 10mm to 50mm in the base of the polygon and along to the antero-posterior axis. The **positive values** on the graph **display the anterior or right side**, the **negative values** display represents **posterior or left side**.
- **Postural instability** : This postural instability displays the evolution of the speed of the body's center of thrust movement during the exam.

Graphs - Postural Analysis

Graphs display options

- **X axis** : For each dynamic analysis graph, you can select to display "Images" or "Temps" (en ms) on the abscissa axis.
- **3D** : 3D Visualization of the graphs.
- **Labels** : Displays the values of each image (to be used in conjunction with the zoom function to facilitate reading the values).
-  : Selecting the thickness of the graph lines.
- **X, Y** : Cartesian coordinates for each point you select on the graph.

Parameters

The numerical parameters are displayed on the right side of the graphs :

- **The Range** of the oscillations evaluates the distance in millimeters between the most anterior point (the farthest on the right) through which the center of thrust ran through and the most posterior point (the farthest on the left).
The range of the oscillations is independent from the positioning of the patient on the platform since the software searches for the extreme points reached by the center of thrust and then calculates the distance between these points along the antero-posterior axis (or latero-lateral axis).
- **The Standard Deviation** measures the scattering in relationship with the average position of the center of thrust. From a pure statistical point of view, 68% of the oscillations have amplitude between +1 and -1 standard deviation, 95% of the oscillations have amplitude between +2 and -2 standard deviations.
- **The average speed** (expressed in mm/sec) along the antero-posterior axis is equivalent to the projection of the entire displacements of the center of thrust on that axis divided by the duration of the recording. Please note the displacements are expressed in absolute value without consideration of the patient direction of movement. A high average speed of oscillations in the sagittal plan may indicate a significant postural instability in the antero-posterior direction.

The global parameters of instability related to patient's balance as described in the foot print analysis (see page 64) are displayed along with the postural instability graph.

Measurements comparison and Romberg quotient

Romberg Quotient is calculated based on the length and surface parameters of 2 specific postural recordings: the first measurement is based on a recording with **eyes open** and the second with the **eyes closed**.


Acquisition


The **recording procedure** is described in the « postural acquisition » paragraph. If you want to perform the test of Romberg, you need to record 2 different postural measurements (in the same exam file).

- ✓ Open the patient's database.
- ✓ Create a new exam.
- ✓ Perform the first recording (eyes open).
- ✓ Perform a second recording (eyes closed).

Analysis

For calculating Romberg's quotient, you need to :

- ✓ Click on the "**Comparison**" tab in the software.
- ✓ Load the "**eyes open**" and "**eyes closed**" recordings.
- ✓ Visualize the quotients "Area" and "Length" by clicking on  (you can also compare postural measurements).

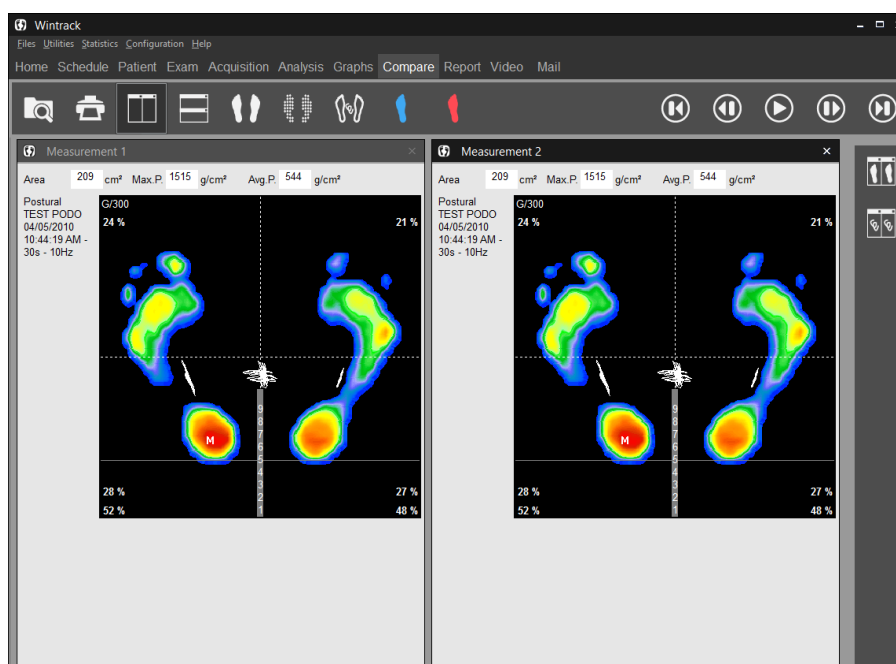


"Comparison" Tab

General information

It allows viewing up to 4 measurements simultaneously on the same screen.



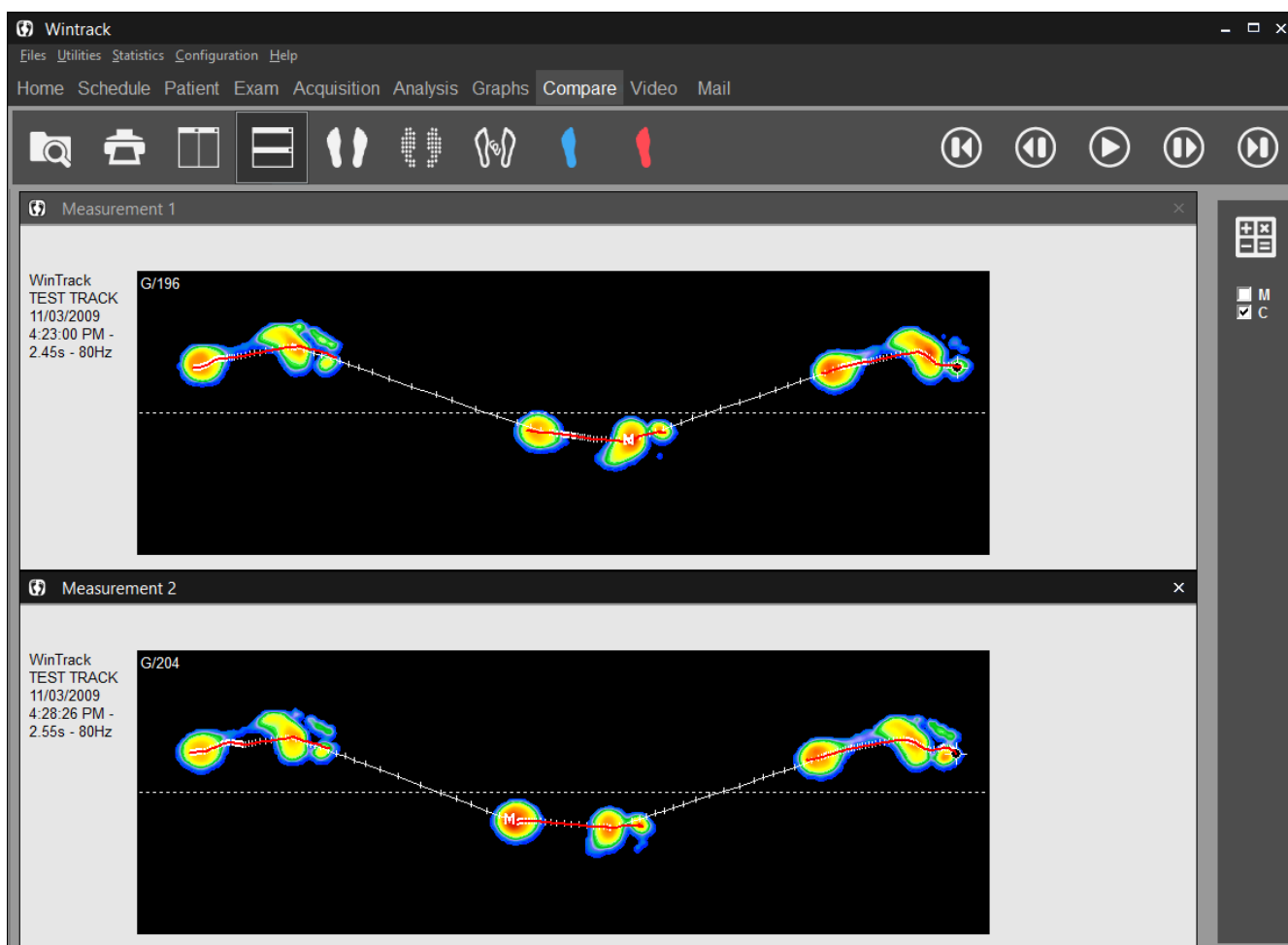

This viewing mode provides the following options :

- **Patient follow up** : viewing measurements on different dates.
- **Comparative study** (Test / Re-Test) : for example, viewing a static measurements without corrective procedure and with correction.
- **Comparison and correlation** of different types of measurements (viewing simultaneously a static, dynamic and postural measurement on the same patient).
- **Comparison between patients.**
- View a measurement in **different modes** of visualization (for example in Standard mode and Isopressure mode).





"Comparison" Tab

Up to 4 measurements comparison

- ✓ Load the measurement to compare by clicking on . Select the measurement in your patient database.
- ✓ Repeat the same procedure for all the measurement to compare (4 max).  and  allows viewing respectively vertical and horizontal mosaic.
- ✓ To unload one measurement, click on  located in the upper right corner of the screen.


Note 1 : the screen not boxed in gray relates to the measurements visualized in the « Detailed analysis » tab".

Note 2 :

- For Static and Postural measurements, you can also compare podometric data by clicking on .
- For Dynamic measurements, you can also compare Space/Time calculations by clicking on .

"Report" Tab

For static or postural measurements, this report printing option provides a practical way to create **"personalized" reports** from different screens available in the program.

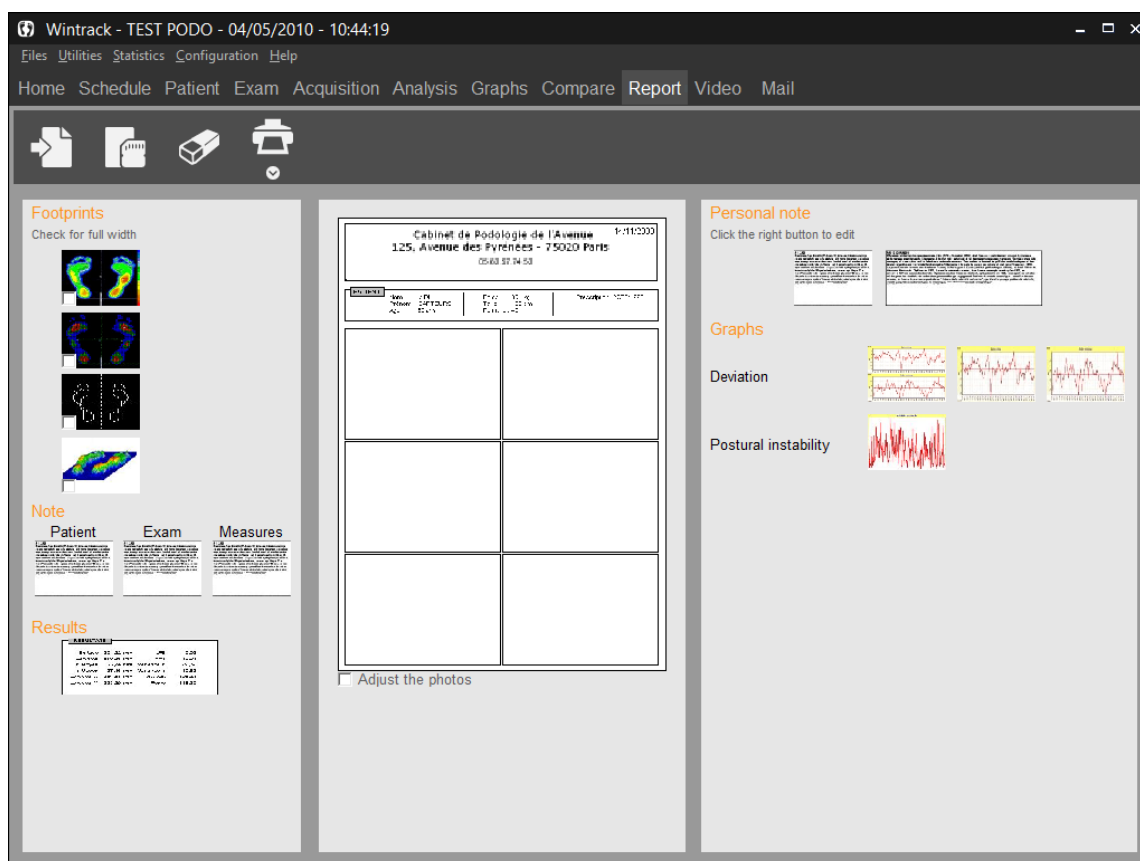

All the available information which can be printed (cartography, graphs, notes, etc.) are located on each side of the central blank page (divided in 6 areas or 3 areas if you select "Full width").

Create your report by moving the information screens you selected from the sides to the central page areas. When your report is created, you can print it directly or save it as a model to use later with another recording.

Tool Bar



Print a report. The arrow gives access to "Attaching documents to an exam" (see page 27).



Save a model report.



Load an existing report model.



Erase the central page to make it blank.

"Report" Tab

Tool Bar


If **all the loaded measurements are of the same type** (Static or Postural) 2 printing modes are available:



Printing of an active measurement.




Printing of all measurements **without preview**.

Select the printing mode by clicking on  to start printing.

Creating a personal note

- ✓ Move/drop the **textbox "Personal Note"** in the location you selected.
- ✓ Right click on the textbox "Personal Note" and select "Edit a personal note".

The following textbox appears :

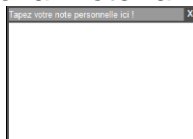

- ✓ Type your text and click on  when finished.

Inserting an image


To insert an image in the report:

- ✓ Right click on an empty box in the report and select "Load image".
- ✓ Search for the image on your computer using the explorer and click on "Open".

It is not possible to mix static, dynamic or postural images on the « pizza » report.

To visualize measurement of different types (for example to compare a static and a postural foot print on a patient), select the "Comparison" tab.

"Video" Tab

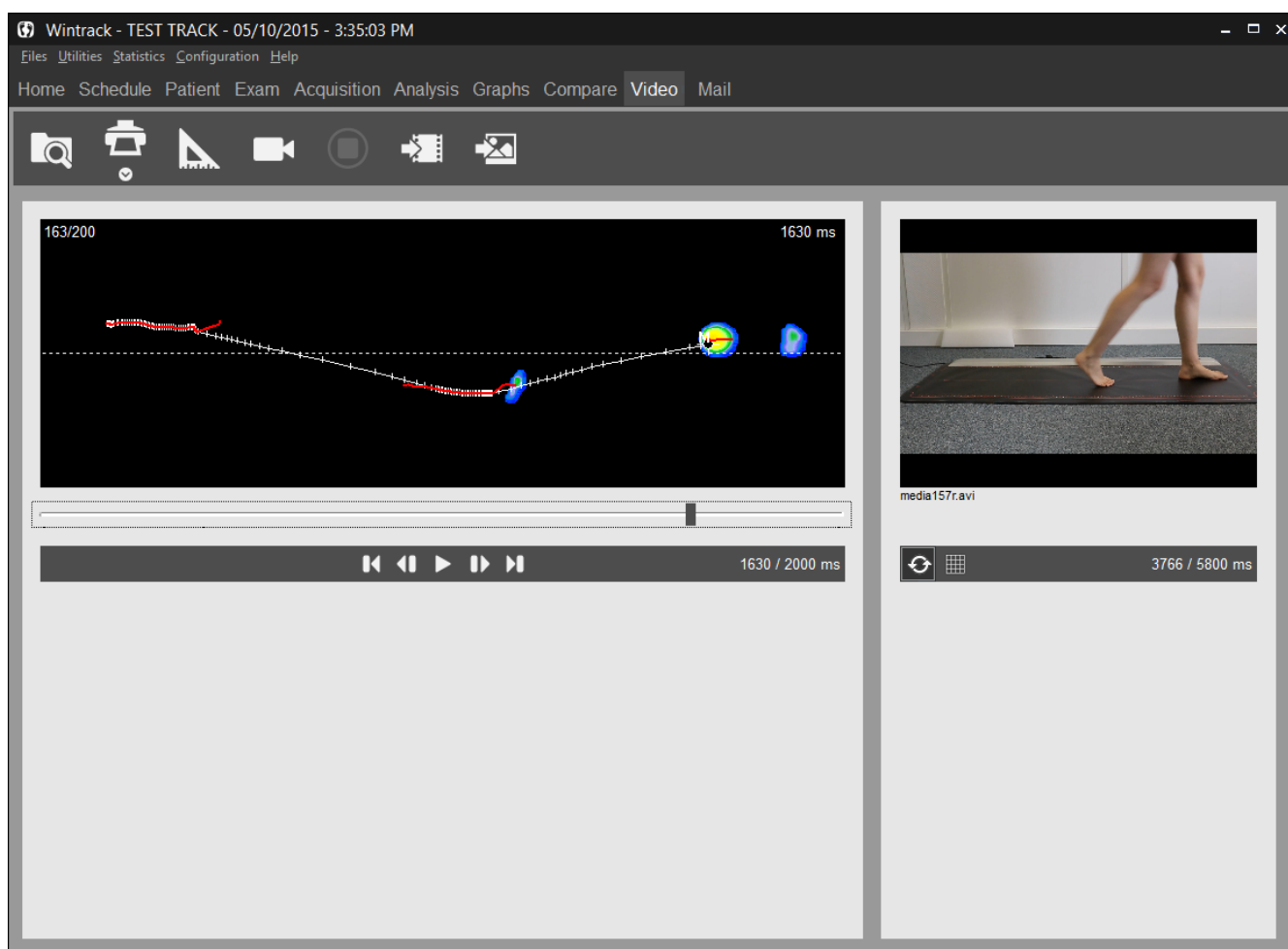

Tool Bar



Print the podometry image and the video image associated. The arrow gives access to "Attaching documents to an exam" (see page 27).



Link an existing video to live exam recording.
This key allows exploring the hard drive to load a video in **asf**, **avi** or **wmv** format.



Link an existing image to live exam recording.
This key allows exploring the hard drive to load an image in **jpg**, **jpeg** or **bmp** format.




Allow recording video at a different time than the podometry acquisition.

"Video" Tab

Simultaneous recording Video / Podometry

Only available with a standard Dynamic acquisition (see page 33).

- ✓ In the "Acquisition" tab, select "Dynamic".

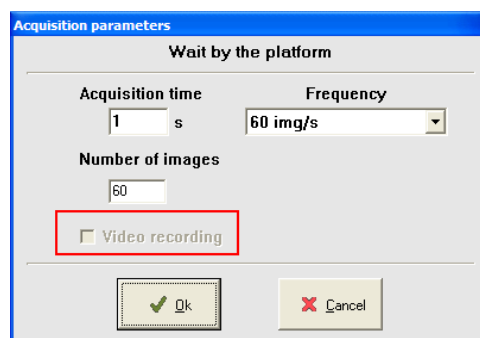






- ✓ Check mark "Video Recording" (not available in "Multiple dynamic" acquisition).

Note : When the "Video Recording" message has a gray background, the computer did not detect the video acquisition system. Verify that the webcam is properly connected and device properly installed.


- ✓ Perform a normal measurement. The software will launch the video acquisition automatically.

Post-synchronization of the video with the dynamic foot print

- ✓ After the video is recorded, it is automatically synchronized with the podometry acquisition. In the "Video" tab, the buttons  and  under the video allow to perfect this synchronization. You must go to the first podometry image to access them and so fix the first video image.
- ✓ If you recorded the video with an other software, you can also synchronize it in the "Video" tab by clicking on right menu . When the two screens are « locked in position », click on .
- ✓ The left menu  then becomes the unique control bar for the 2 animations.



"Mailing" Tab

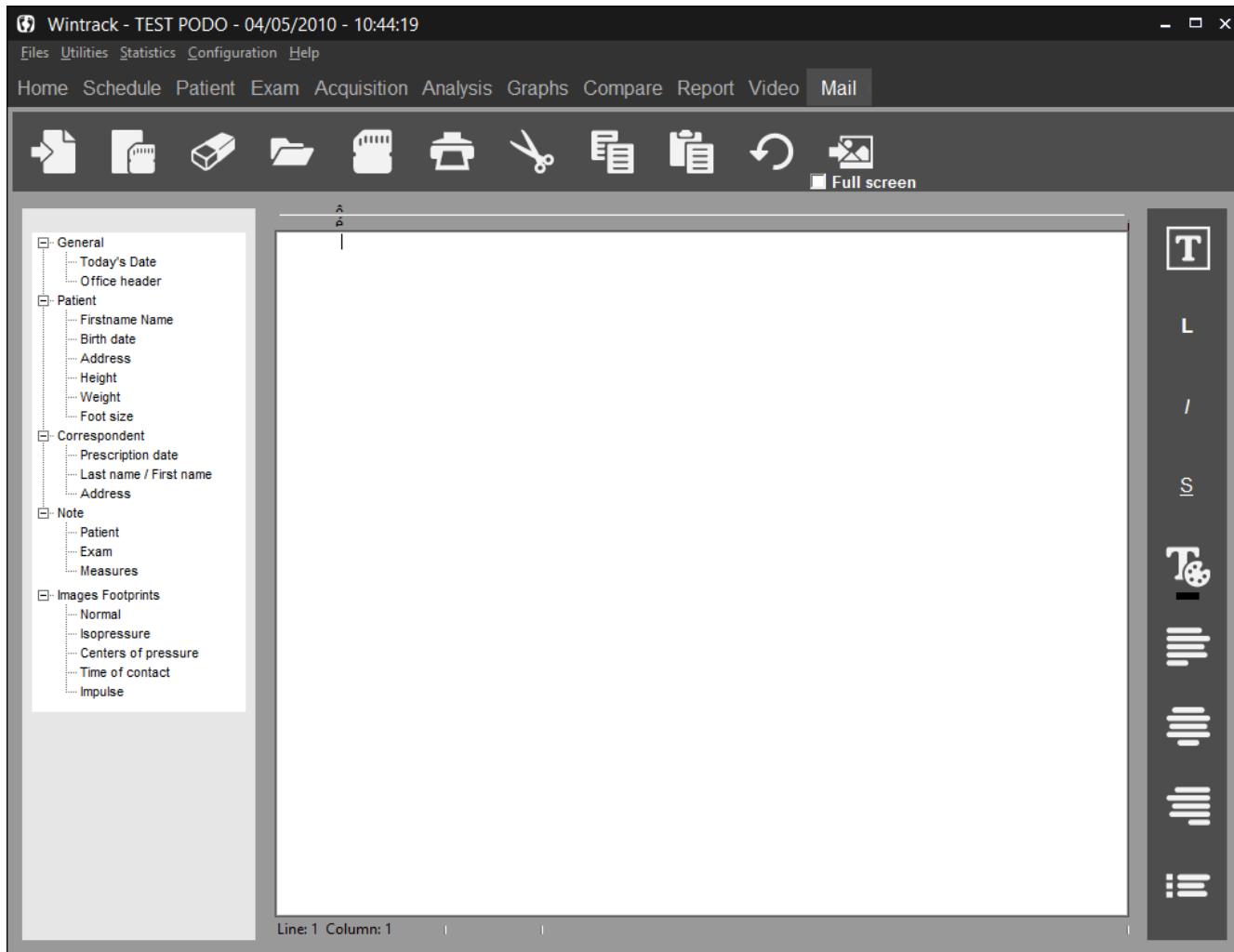Allow to generate model letters to attach with your reports.



On the left side of the screen, you have access to the information categories from the patient file, exam and feet images.

To compose your letter, type your text and then double click on the line or image you wish to include in the letter.

You can also save a model letter by clicking on . Then you will only have to reload it by clicking on  and complete it with the specific patient informations.

"Mailing" Tab

Tool Bar



Search for an existing model letter.



Save model letter.



Erase the page.



Open an old patient letter.



Save the letter in txt format (to export in Word).



Print letter.

To format the letter, you have the following functions :



Cut the selected text.



Copy the selected text.



Paste the selected text.



Undo the last modification.



Change the font.



Put respectively in bold, italic or underlined the selected text.



Change the selected text color.



Justify the selected text respectively on left, center or right side.




Add chips.



Insert an image. By clicking on "Full screen", the image will occupy the width of the page.

Printing

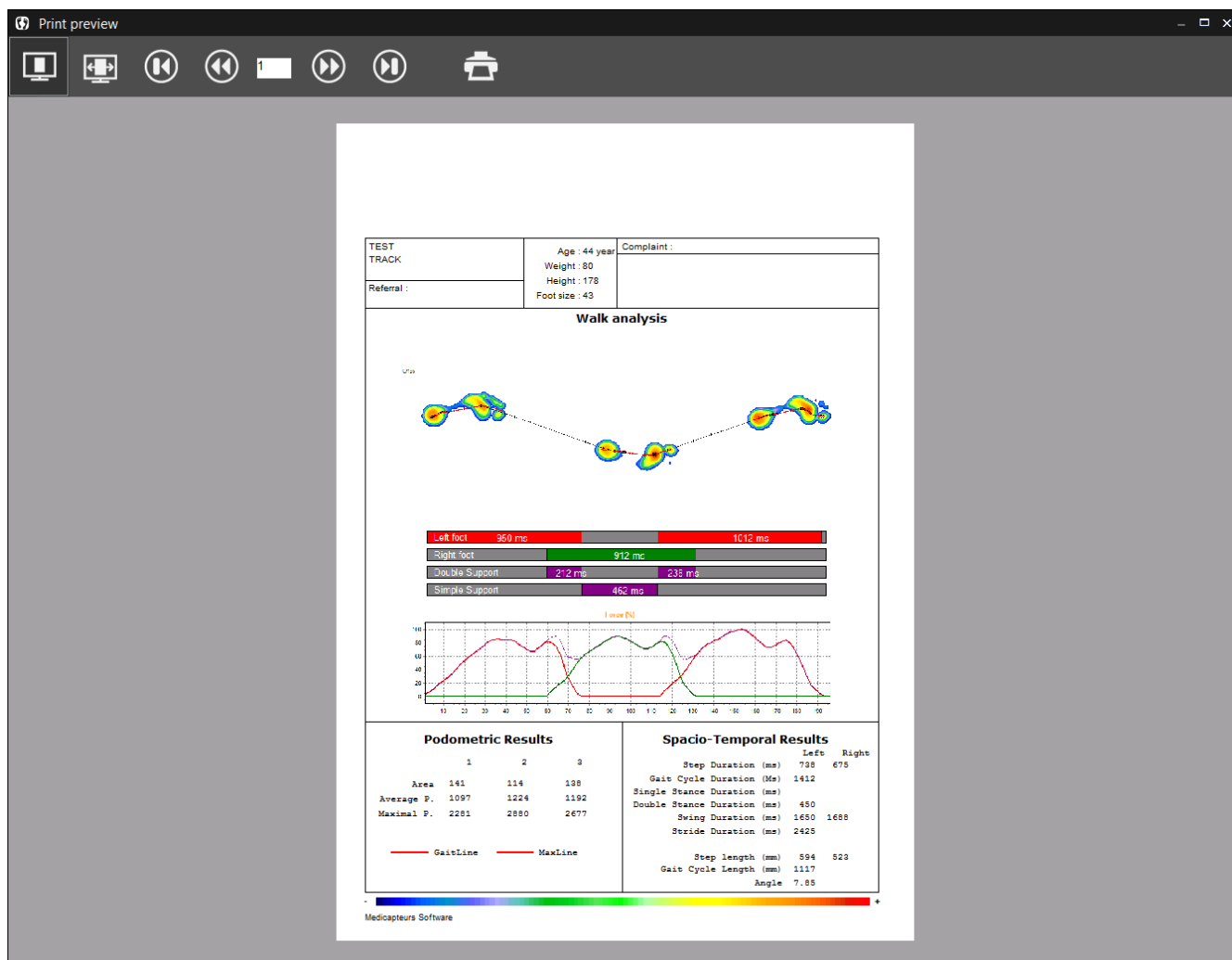

Tool Bar




Click on this key to access print menu.

Full page print preview.

View full screen preview.


Print.

You can remove the print preview and printing will be launched when you click on  in one of the tabs in the software. Go to *Configuration/Parameters/General* (see page 16).

Note : You can download and install pdf Creator or FOXIT, free softwares which appear as a printer and allow to create PDF format files.

Scale 1 Printing

In the "**Detailed Analysis**" section, you can print the foot pressure mapping in **real size** (within one captor resolution level).

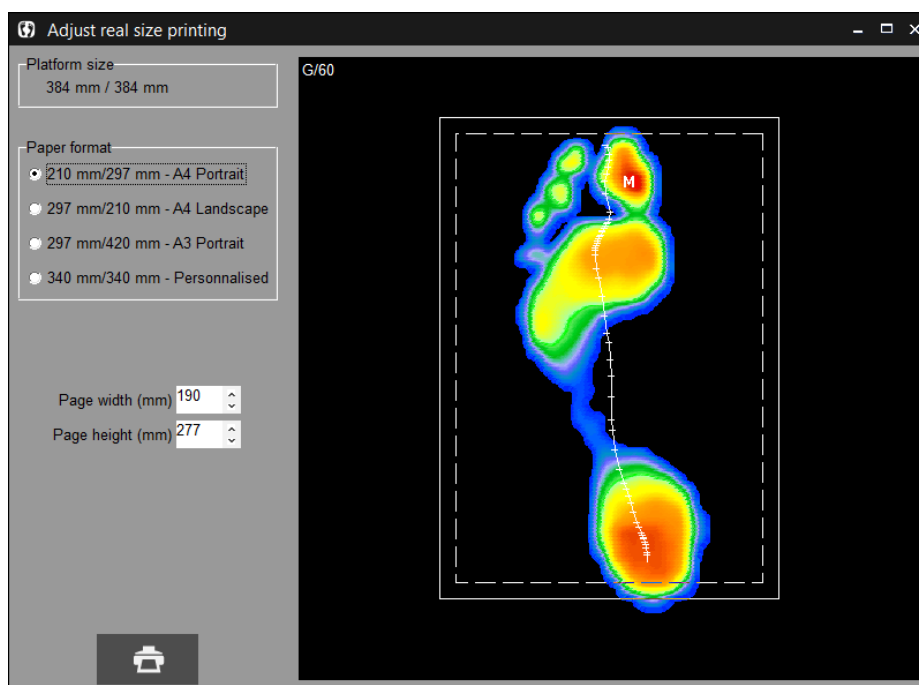

Click on the arrow  and select the menu "Print real size".

The "Centering of the real size footprint" appears:

This particular screen allows **defining the area to print**. In fact the regular paper size is smaller than the active surface of the plate ; therefore it is necessary to select the portion of the cartography you want to print.


Usually, you print one foot after the other.

The "**Format Paper**" window allows selecting a box which is superposed to the cartography. This box coincides with the printable area which is properly displayed according to your paper size.

To position the box on the area to be printed, position the mouse inside the box. Left click and press on the mouse and move the box to the selected area.

Note 1 : You can reduce the pre selected "width" and "height" by clicking on the arrows .


Note 2 : If you have a printer with A3 paper, you can print the 2 feet at the same time.


When the box is properly positioned on the footprint, click on  to access to the printing menu (see previous page).

Exporting Data

By right clicking with the mouse on the “detailed analysis” tab or on a graph in the “graphs” tab, you can:

- **Copy in the clipboard** the selected image.
- **Save on a database** the selected image in Bitmap format. A window appears to target the file name and the folder.
- **Export a video** allow to create an AVI video of a dynamic or postural acquisition. A window appears to specify the name and the folder of the created video.
- **Exporting series of recordings to Excel.** The window appears :

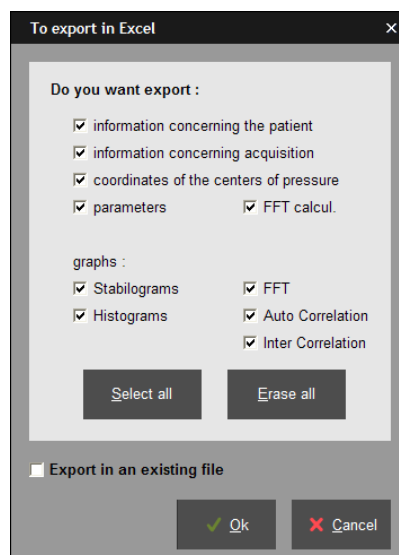

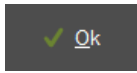- ✓ Select the information to be exported.
 - ✓ If you have to export in an existing file, click on the corresponding check box.
 - ✓ Click on . An excel file is open including multiple pages where each type of available information is presented.
 - ✓ Record the excel file on the hard drive.
- **Exporting matrix** (in the “detailed analysis” tab). It allows creating a text file which contains all the captors’ numeric values. A window appears to specify the name and location of the file that is getting created.
 - **Full page, other graph** or **2 graphs** (in the "Graphs" tab) allow to display graphs on a full page or to return to 2 graphs visualization.

Patient database back up

WARNING!

It is necessary to frequently backup the "WIN-TRACK Software" patient database which includes all patients. In fact, a technical computer dysfunction may damage, even destroy all your data which then cannot be retrieved.

One back up per week is recommended.

Patient database back up

Copy the folders **Data**, **Documents** and **Movies** located in **C:\win-pod** to a back up unit, such as :

- CD or DVD writer
- ZIP Drive
- Another Hard drive
- USB Key

Reloading a backed up patient database

If you are not certain of the procedure, please contact us.

Procedure to follow when installing the back up database on a new computer configuration:

- ✓ Install "WIN-TRACK Software" following the instruction manual.
- ✓ Launch the DBManager utility function which can be accessed through **Start/All programs/ Medicapteurs**.
- ✓ In the DBManager section :
 - Click on the command "**Import a database**".
 - The following window opens "Assistant for importing a database".
 - Select your back up unit.
 - Double-click on your back up DATA folder.
 - Then click on "**Next**" and then on "**Start**".
- ✓ When the import is completed all the patients are directly integrated in the patient database of the "WIN-TRACK Software". You can verify that importation was successfully completed by opening the "WIN-TRACK Software" and then by clicking on "search" in the "Patient" tab.
Verify then the displayed list and that the associated measurements are present.

COPYRIGHT

Copies or reproductions of this document is strictly limited to private use and not authorize for public or collective use, and excerpts or short quotes from this document need to clearly include the author and the source.

All rights reserved. Duplication of any part of the document or the entire document without express written consent of Medicapteurs SAS is illegal and strictly prohibited.

© MEDICAPTEURS - Toulouse - FRANCE - November 2015

Notes

[illegible]

Notes

[illegible]



« Les Espaces de Balma »
18 avenue Charles de Gaulle - Bât. 34 - 31130 BALMA
Tel :33 (0) 562 571 571
www.medicapteurs.fr

CE 0124
Copyright 2015