





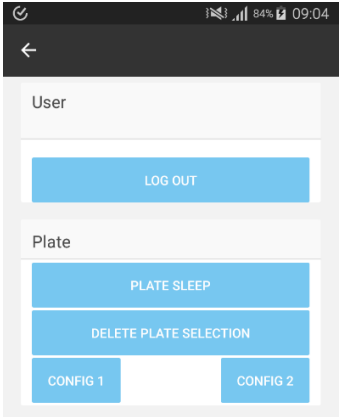


PodoCapture – Quickstart

<p style="text-align: center;"> First Acquisition</p>	<ol style="list-style-type: none">1) Connect manually to the platform from the mobile wifi manager by selecting your platform (xxxx_SVR_xxx) in wifi networks.2) Open PodoCapture and create an account if you deem it necessary.3) Create a new patient. Enter the weight to display the calculations.4) Choose the type of measurement.5) At the first acquisition PodoCapture will need to register your platform. The mobile wifi manager opens, check that the platform is selected and click on the mobile back button or  to return to PodoCapture. The software will automatically select this network for future acquisitions.6)  Allows you to modify the parameters of acquisition in dynamics and posturology.7) Click the button  to make the acquisition.
<p style="text-align: center;"> Inactive acquisition button Settings screen, accessible from the Patients screen with the button  :</p> 	<ul style="list-style-type: none">➤ The platform is not turned on.➤ The selected wifi (step 1 above) is not the one of the platform. This setting can be reset in the setup screen with the "Delete platform selection" button.➤ The platform is already connected to another device (the PC for example) and therefore can not connect to the mobile. A central light on the hood of the lit and fixed platform indicates an established wifi connection.➤ The mobile has switched to another wifi network.➤ If the correct wireless network is selected, go back to the patient screen and make a new acquisition.➤ Restart the platform.➤ Turn off the wifi on the mobile and let PodoCapture reconnect it.➤ If you have never been able to acquire and all previous points have been verified, please change the configuration of PodoCapture using the "Config 1" button and retry an acquisition. Do the same with "Config 2" if necessary.