
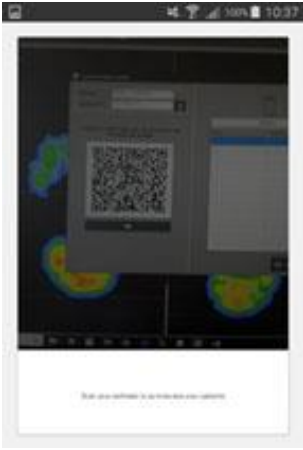
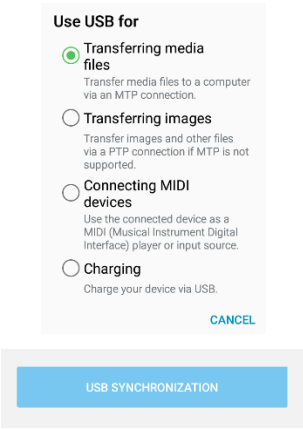

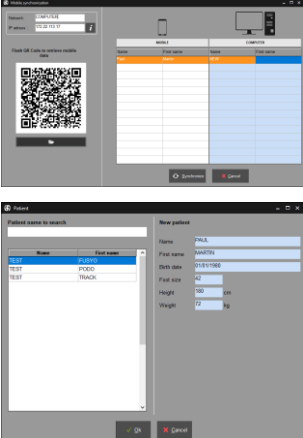


PodoCapture – Unloading measures

<div style="text-align: center;">  <p>Unloading measurements on the PC</p> </div>	<p>Accessible from the Patients screen of PodoCapture.</p>
<p>Unloading by wifi (ios)</p> 	<p>On the PC:</p> <ol style="list-style-type: none"> 1) Connect to a wifi network (box, etc.) 2) Open the software, go to the menu "Utility -> Mobile Synchronization". 3) Check that the "Network" and "IP Address" fields are correctly filled in. <p>On the mobile:</p> <ol style="list-style-type: none"> 1) Connect to the same wifi network. 2) Scan the QR Code displayed on the PC.
<p>Unloading by usb (android)</p> 	<p>On the mobile:</p> <ol style="list-style-type: none"> 1) Connect the mobile to usb on the PC. 2) Switch the mobile to "Transfer media files" mode. 3) Click the "USB Synchronization" button. <p>On the PC:</p> <ol style="list-style-type: none"> 1) Open the software, go to the menu "Utility -> Mobile Synchronization". 2) Click on the button  and look for the file Synch.json on the mobile in the folder "Download / Podocapture" or "Download / Podocapture".
<p>Synchronization on the PC</p> 	<p>On the PC, once the unloading is finished (by wifi or usb), in the "Mobile synchronization" window:</p> <ol style="list-style-type: none"> 1) You can double-click on each loaded patient if there is an assignment error to open the patient file and correct mis-assignments. For a new patient, you can complete the fields Size, Height and Weight if you have not done it in PodoCapture. 2) When everything is correct, click on "Synchronize". 3) Once synchronization is complete, this screen closes automatically and you can use your software as usual.